

USER INSTRUCTIONS

HOME HELPER TROLLEY - GREY

S34664

Home Helper Trolley - Grey

⚠ General Safety Warning ⚠

Read and follow these instructions carefully.
Keep these instructions in a safe place for future reference.

1. INTENDED USE

To help users to safely transfer items and meals around the home. It's lower tray is set forward on the frame to allow users to walk naturally when pushing.



3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

2. SAFETY



WARNINGS

Take care when moving across thresholds.

Do not:

- Use this trolley as a general walking aid and do not sit on it.
- Overload the trays. Maximum tray capacity is 5 kg each tray. Position items as centrally as possible.
- Use the trolley outdoors.
- Use the trolley to assist when attempting to stand from a seated position.


3.1 Contents:

1. Frame Side Right x 1
2. Frame Side Left x 1
3. Tray x 2
4. Cross Tube Front x 2
5. Cross Tube Rear x 2
6. E-Clip x 4
7. Castor Assembly x 4



4. ASSEMBLY/INSTALLATION

You do not need tools to assemble the trolley.

4.1 Fitting the Cross Tubes

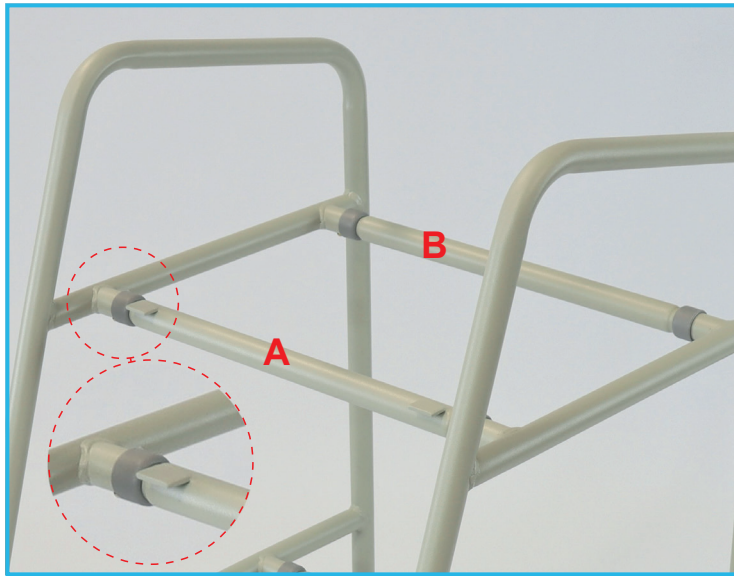

The trolley has four cross tubes, two front tubes (A) and two rear tubes (B). The front tubes have welded tabs to hold the fronts of the trays. Fit the front tubes so their tabs face forward and upwards.

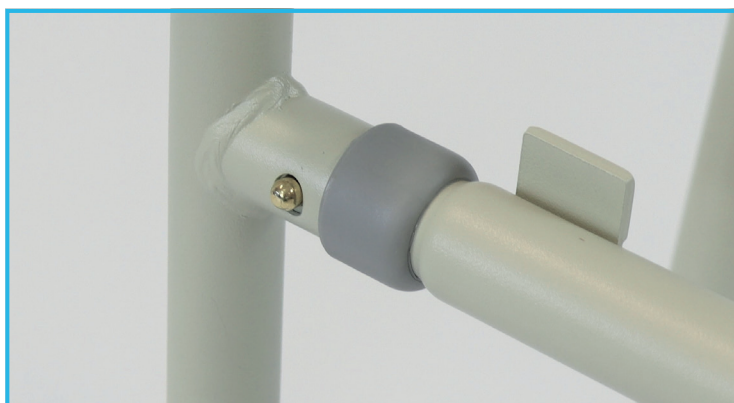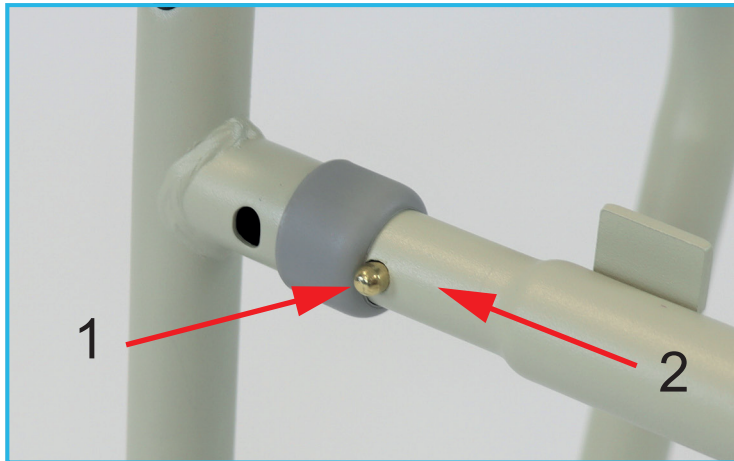


To engage the tubes in the frame, press in the spring button (1) and push in the tube (2) until the spring button 'clicks' into position and the button is visible through the hole in the frame.

4.2 Fit the Other Frame

Align the other frame side onto the four side tubes and engage the spring buttons as described above, ensure all spring buttons have clicked into place and are visible through the holes in the frame.

4.3 Fitting the Castors

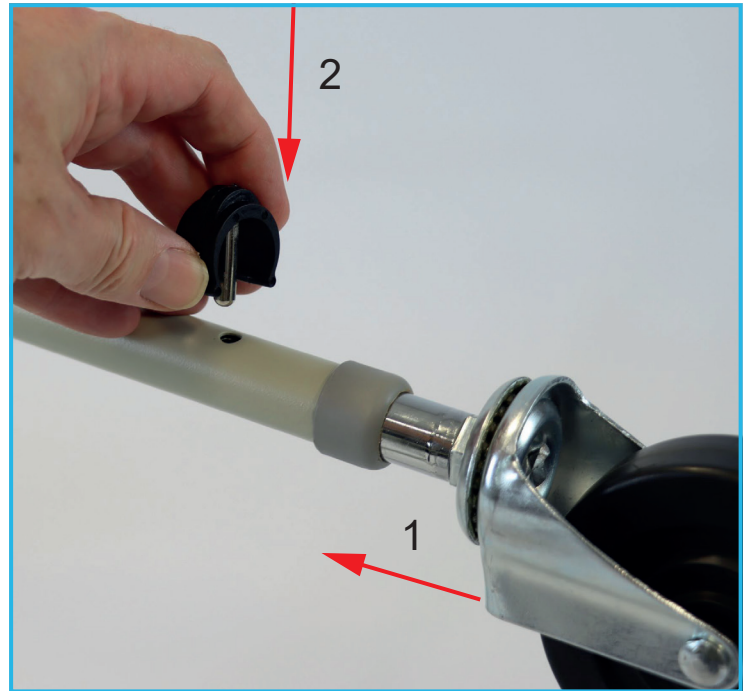

1. Push each castor into a hole at the bottom of the frame, aligning the holes in the frame with the holes in the castor leg.
 2. Secure each castor with an E-clip.
- Note:** Set the castors to the same height.
3. Turn product upright and check it is level and stable.





4.4 Fitting the Trays to the Frame



Note: Fit the lower tray first.

1. Hook the front of the tray into the tabs on the front cross tubes.
2. Clip the rear edge of the tray onto the rear tube and press down to lock it into position.
3. Repeat for the upper tray.
4. Check that the trays are level before use.

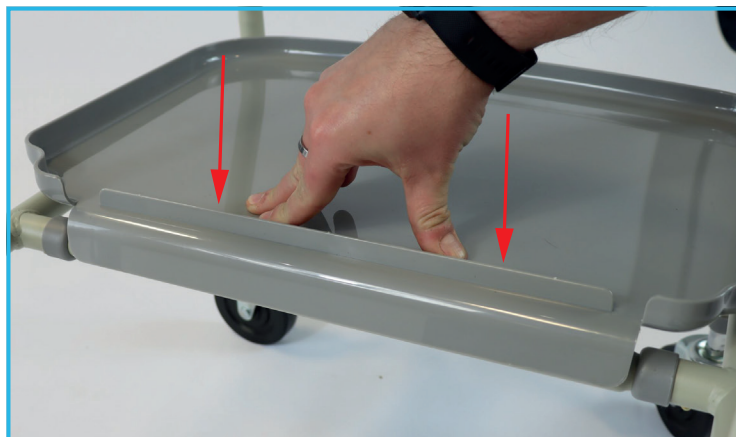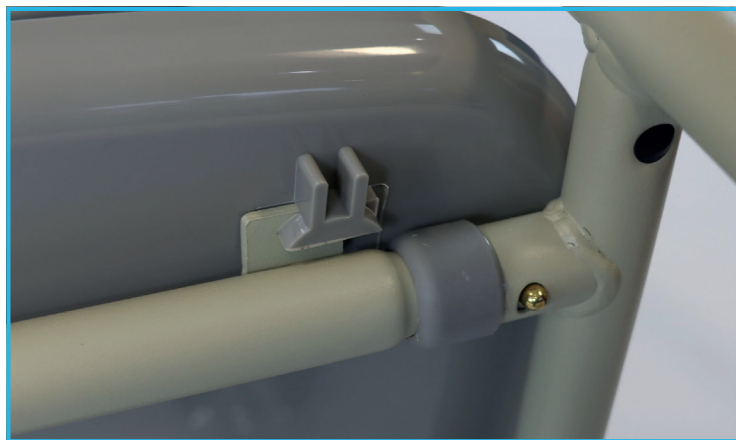

5. MAINTENANCE

Inspect the product regularly for signs of wear or damage, particularly the condition of the castors.

Check all fixing clips are inserted correctly and that there is no excessive movement.

Do not use if you see or suspect a fault – contact your dealer immediately.

6. CLEANING INSTRUCTIONS

Clean regularly using a general purpose neutral detergent and warm water.

Do not use abrasive cleaners or cloths as this will damage the surfaces of the product.

Allow to dry thoroughly.

The trays easily clip off and on for cleaning. To remove, unclip the rear edge and lift clear. To refit, refer to Section 4.4.

7. ACCESSORIES & SPARE PARTS

- S28792 - Home Helper Trolley Spare Tray - Grey.

For more details please visit our website or call Customer Services on the details below.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors and Recycling Centres.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111
Email: customerservice@nrshealthcare.co.uk

EC REP

NRS HEALTHCARE 4C Altona Road, Blaris Industrial Estate, Lisburn, Co Antrim, Northern Ireland BT27 5QB

Email: exportsales@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk