

# User Instructions

## M0122 Cordless Kettle Tipper

### General safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### 1.0 Intended Use

The Kettle Tipper is an aid to help take the strain off your wrists and to avoid the need to lift heavy kettles full of water. It is suitable for most jug kettles of about 1 to 2 litres.

### 2.0 Safety Precautions

Ensure kettle is securely fastened in tipper before use.


It is advised to always leave the kettle on the tipper and use a small jug to fill with water to a recommended maximum of  $\frac{3}{4}$  full.

Position the kettle tipper away from sources of heat.

Take care to keep fluids away from electrical sources.


### 3.0 Unpacking

Remove all packaging and check the product is not damaged. Any suspected faults should immediately be reported to the supplier and the product taken out of action.

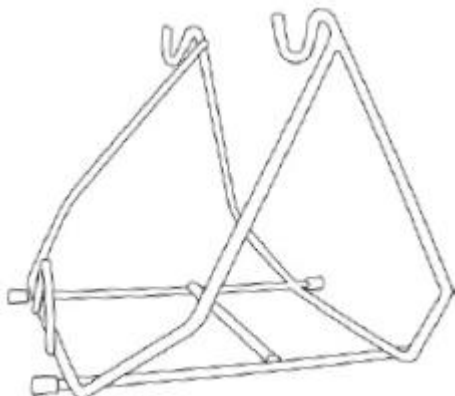


### 4.0 Installation


Fit the tilt plate to the main framework by hooking over the frame.

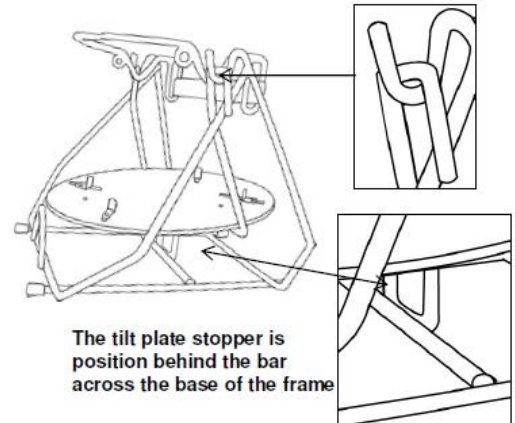

Secure the kettle base onto the tilt plate using the clamps and wing nuts provided.

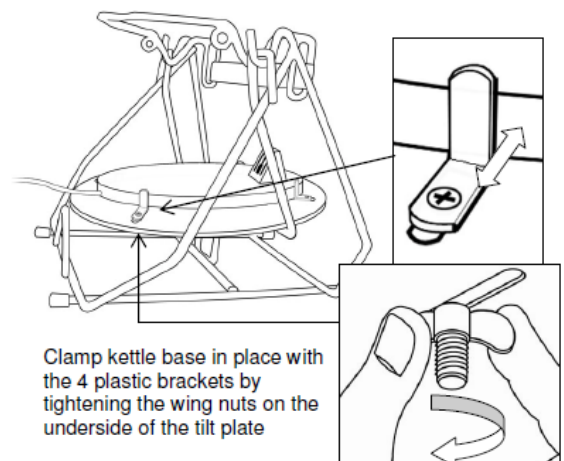

Place the kettle onto the tipper and fasten in position by attaching the Velcro strap.

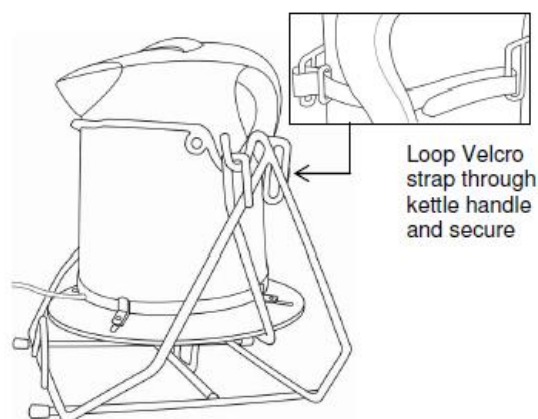

## 5.0 Use

### ***Filling the Kettle***

It is advised to leave the kettle permanently in the tipper. The kettle can be filled by using a jug to take the water from the sink to the kettle. This avoids carrying a heavy kettle full of water from the sink. Always make sure the element is covered but to avoid any potential spillages, we recommend that the jug kettle be filled no more than three quarters full.

### ***Pouring***

Put the cup or mug etc. that you wish to fill in front of the spout against the 'A', at the front base of the frame. Pour from the kettle by tipping it gently forwards (see image).



### **NRS Healthcare**

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

## 6.0 Cleaning

Ensure the mains electric supply is unplugged prior to cleaning the kettle tipper.

Clean regularly using a general purpose, neutral detergent and warm water.

Do not use abrasive cleaners or cloths as these could permanently damage the product.

To remove the kettle for descaling, remove the Velcro strap and lift off the tipper.

## 7.0 Maintenance

Check for general wear or damage. Any suspected faults should be reported to supplier immediately and the product taken out of service.

## 8.0 Product Disposal

Product is made from plastic and metal and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 9.0 Product Guarantee

NRS Healthcare, guarantee this product for a period of 12 months from the date of purchase against faulty materials and workmanship. This guarantee does not cover, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

Tel. Head Office: 0330 100 0098

Tel. Customer Services: 0345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**