


N72802 – Seahorse Plus Lap-Strap Small

Fitting instructions and instructions for use




Fitting the Lap-strap

The lap-strap should be fitted to the chair before use. Please follow the instructions below:

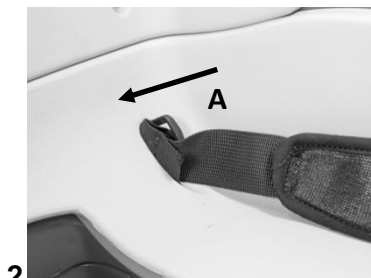

- 1 The lap-strap is fitted by feeding the two rear buckles (A) through the slots in the seat liner and seat. The Seahorse label should be on the left as viewed from the front
- 2 The buckles (A) must be pushed fully through the slots, by rotating them through 90 degrees
- 3 The straps should then be pulled from the front, to locate the buckles flush at the rear of the seat. Make sure the buckles cannot pull through the seat, by tugging firmly

Using and Adjusting the Lap-Strap

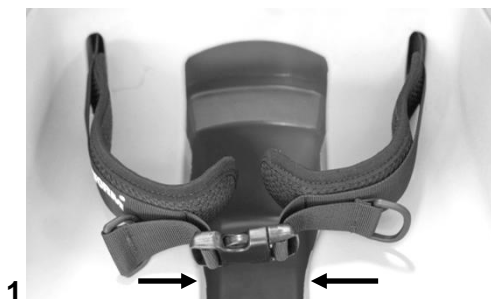

- 1 Position the user in the chair and buckle the lap-strap loosely around the users waist
- 2 Pull the two D-shaped toggles at the front to tighten the lap-strap

WARNING: Ensure the lap-strap is correctly adjusted for the user, and is a snug fit. Incorrectly adjusted lap-straps can result in the user sliding forward in the chair.

Issue 2 17.06.16