

## Product Technical Data Sheet 043

### 2 in 1 Contour Community Bed Rail

#### Product Range

N72000	2 in 1 Contour Community Bed Rail
--------	-----------------------------------

#### Product Description & Intended Purpose

The Community Bed Rail provides a stable support to assist clients getting in and out of bed it can also be used to aid repositioning when lying in bed.

This device is designed to be fitted to profiling metal framed community beds with 50mm x 25mm (2"x 1") rectangular side tubes.

#### Product Features

- Suitable for use with community type, metal profiling beds
- Moulded hand rail offers enhanced safety for the user.
- Multiple hand grip options for extra comfort.



#### Technical Specification

##### Dimensions (mm):

- Width: 335 (13")
- Height from bed: 562 (22")

**Maximum User Weight:** 160kg (25st)

#### Installation Information: *(Note: for detailed information refer to User Instructions)*

- This product is delivered flat pack and requires assembly before use.
- Assembly time is approximately 5 minutes.
- An Allen key is provided to assist with assembly.
- It can be fitted to either side of the bed depending upon the needs of the user.
- The hand rail can be removed without the need for the bracket to be removed also.

## PTDS043

### 2 in 1 Contour Community Bed Rail

#### Safety Precautions

- An individual risk assessment must be carried out for each user of the bed rail.
- To avoid possible head entrapment, position the Bed Rail at least 320mm (12.5") away from both headboard and footboard.
- Do not exceed the maximum user weight of the product 25st, 160kg.
- Always check the profiling operation of the bed with the bed rail already fixed in place for any potential risks prior to use.
- Take care on beds with rising backrests. In some cases when the backrest raises the gap between the bed rail and bed could become less than 25mm.
- To prevent hand entrapment do not hold the hand rail when altering the profile of the bed.

#### Cleaning

- Clean regularly, using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water. Do not use bleach to wash the straps.
- Always allow product to dry thoroughly prior to use or storage.

#### Maintenance

- Check the 'e-clips' are inserted correctly and that there is no excessive movement.
- Check the hand wheels and grub screws are tight and the bed rail is securely fitted to the bed frame.
- Inspect the product regularly for signs of wear or damage or if the bed rail is loose. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

#### Accessories / Related Products

None

#### FAQ's

None

#### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK