

# Configura Advance External Lateral Support Instruction Manual

IFU-SC1-005EN Rev 01

March 2021

## Contents

Title	Page
Welcome	2
1. Technical Specification	3
2. Assembly	3
3. Setup	3
4. Functionality Check	3
5. Cleaning and Disinfection	3
6. Guarantee	3
7. Disposal	3
8. Contact Details	4

## Design Policy and Copyright

® And ™ are trademarks belonging to Accora Ltd unless otherwise stated. As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. © Accora Ltd 2020.

## Welcome

Dear Customer,

Thank you for purchasing an Accora healthcare product. We feel sure that this product will exceed your expectations.

This manual is important for understanding how to use your new Configura Advance External Lateral Support; please read it before use. If you need further information, please contact Accora.

See Contact Details in Section 8.

Before using the Configura Advance External Lateral Support, you must read and understand the instructions in this user manual. All actions and handling of the Configura Advance External Lateral Support must be performed in accordance with the instructions in this manual.

Any actions that are inconsistent with the manual are performed at your own risk and Accora shall not be liable for any injury or damage. Please ensure that the manual is available to users and operators throughout this products service life.

This instruction manual must be read in conjunction with the Configura Advance chair instruction manual (IFU-SC1-001EN or IFU-SC3-001US).

## WARNING

A clinical assessment and risk assessment **MUST** be carried out by suitably trained and qualified personnel before fitting this accessory to the chair.

Ensure the patient is not at risk of sliding down the chair and becoming trapped by the External Lateral Support.

Refer to the Configura Advance chair manual (IFU-SC1-001EN or IFU-SC3-001US) for full list of safety warnings.

## 1. Technical Specification

The table below lists the part number for the Configura Advance External Lateral Support.

Part number	Description
EXTLAT-0-SC1-020	Advance External Laterals

The External Lateral Support may be fitted to the following Advance chair models:

Part number	Description
CHAIR-0-SC1-030	Advance chair, 100mm castors, standard backrest, manual
CHAIR-0-SC2-030	Advance chair, 100mm castors, standard backrest, electric
CHAIR-0-SC3-030	Advance chair, 125/150mm castors, standard backrest, manual
CHAIR-0-SC4-030	Advance chair, 125/150mm castors, standard backrest, electric
BRPOSKT-0-SC1-030	Configura Advance Postural Backrest

## 2. Assembly

1. Check that the delivery is completed and whether any visible damage has occurred to the accessory kit during transport.
2. The Configura Advance External Lateral Support should be supplied with following (Figure 1):
  - a. Left-hand side support.
  - b. Right-hand side support.
  - c. Adjust slider x 2EA.

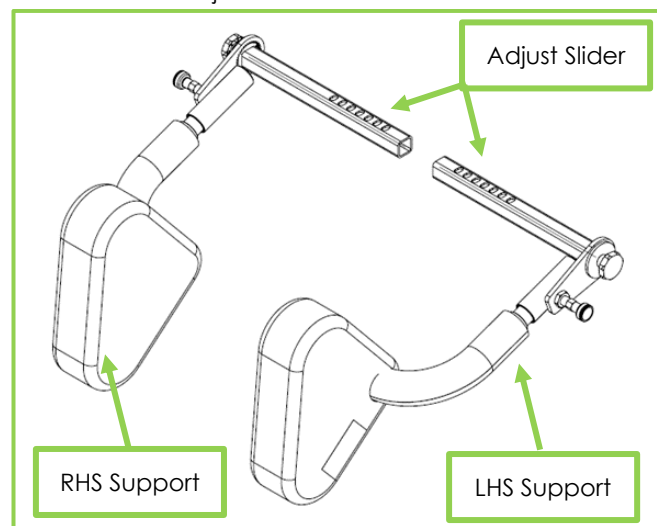


Figure 1

3. Undo the thumb screws shown in red (Figure 2, Item 1) counter-clockwise, insert the adjust slider into the square tube on backrest frame to the desired position, tighten the thumb screws to secure the lateral support position.
4. Lift the lock plunger (Figure 3, Item 1) and fit the left- and right- hand side lateral support into the

slide tube. Release the plunger to lock the support firmly.

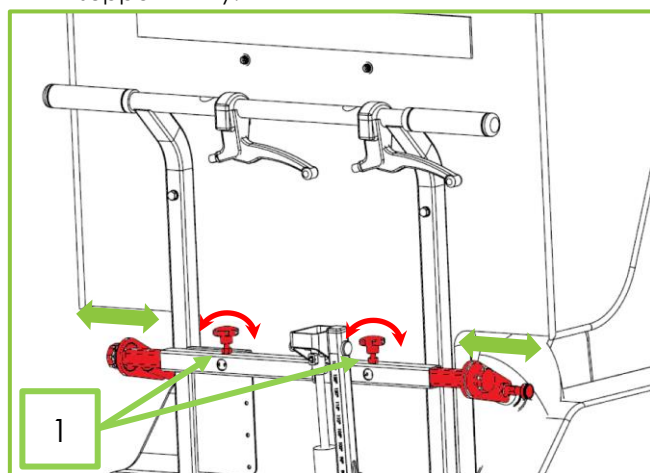


Figure 2

5. Check both left- and right- lateral supports have been tightened and are secure.

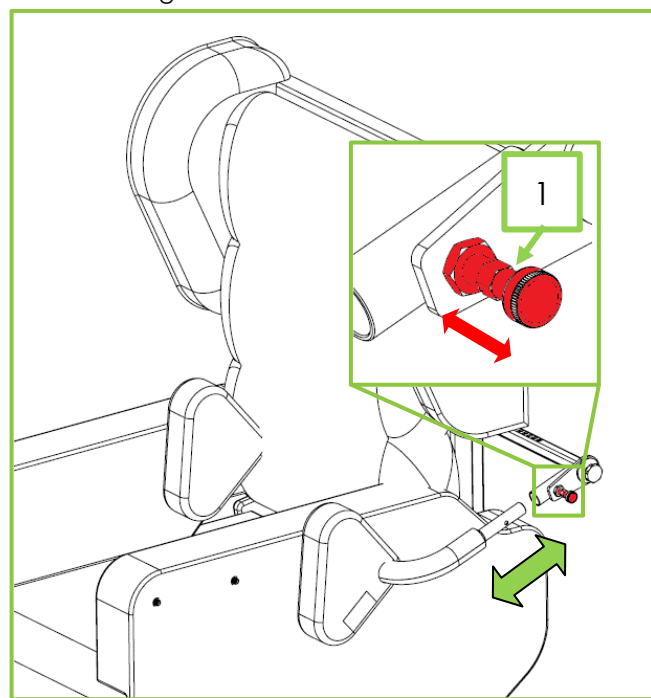


Figure 3

## 3. Setup

1. **Width of the external laterals** – turn the thumb screws shown in red (Figure 2, Item 1) counter-clockwise to release the external laterals. Adjust the width as required, then tighten the thumb screws (Figure 2, Item 1).
2. **Angle of the external laterals** – turn the thumb screws shown in red (Figure 4, Item 1- there is a second screw hidden from view on the other side of the backrest) to release the external laterals. Move the arm away from the sliding bar and adjust the angle as required. Then locate the arm on to the sliding bar again and tighten the thumb screws.

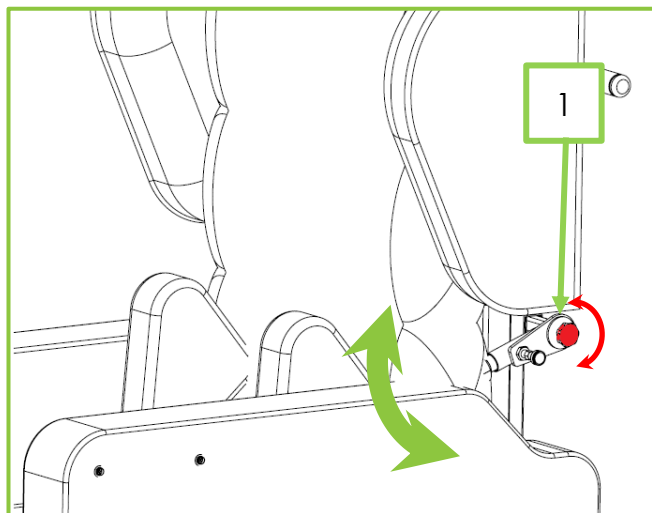


Figure 4

3. **Removing the external laterals** – lift the lock plunger shown in red in Figure 3 and remove the external laterals. To refit, lift the lock plunger and slide the external laterals into position. Release the lock plunger and move the external lateral in and out slightly to ensure the lock plunger engages.

#### 4. Functionality Check

##### **WARNING**

Functionality check **MUST** be carried out by suitably trained and qualified personnel.

Always engage the castor brakes when the chair is stationary or left unattended.

These checks must be carried out after the External Lateral Support has been adjusted, or if any other adjustments have been made to the chair.

1. Ensure the External Lateral Support is positioned correctly for the patient and locked by the hand screws.
2. Check that the tilt angle of the External Lateral Support is firmly tightened by the hand screws.
3. Check that the External Lateral Support is engaged by the plunger, with no un-wanted rotation and movement.

#### 5. Cleaning and Disinfection

##### **WARNING**

The chair must be disconnected from the power supply when being cleaned or disinfected.

All functions **MUST** be tested and approved by a competent person after cleaning or disinfection.

The chair **MUST** be cleaned and disinfected before re-using the chair for a different patient.

##### **Cleaning Information:**

Clean by wiping with a damp cloth. Alternatively use Microfibre cleaning cloth without any soap.

For extreme soiling use a 5% Sodium Hypochlorite solution.

The External Lateral Support is not designed for maintenance in automatic chair washers or for cleaning with pressurised water, spraying, showering or steam cleaning.

Accora cannot be liable for any damage or risk of damage if inappropriate cleaning or disinfectant agents are used.

#### 6. Guarantee

Model	Configura Advance External Lateral Support
Warranty period	2 years

#### 7. Disposal

In the event of the disposal of materials from the chair, end-of-life parts must be disposed of in accordance with current environmental regulations.

## 8. Contact Details

	UK and Rest of World	USA
Address	Accora Ltd. Charter House Barrington Road Orwell Cambridge SG8 5QP UK	Accora Inc. 9210 Corporate Blvd. Suite 120 Rockville MD 20850 USA
Telephone	+44 (0)1223 206100	+1 301-560-2400
Email	info@accora.care	information@accora.care
Website	www.accora.care	www.accora.care

*This page is intentionally left blank*

*This page is intentionally left blank*

# Accora



Accora  
Barrington Road  
Orwell  
Cambridge  
SG8 5QP  
United Kingdom

T: +44 (0)1223 206100  
[info@accora.care](mailto:info@accora.care)  
[www.accora.care](http://www.accora.care)