

USER INSTRUCTIONS

NRS HANDLING SLING

L97779

NRS Handling Sling

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

To extend the carer's reach during a variety of moving and handling procedures – It is particularly useful to assist a person to move to the edge of a chair before standing, also for repositioning in bed where support around the torso is required. The soft lining is gentle on damaged and delicate skin whilst the dual handles provide flexibility and carer comfort.

Leg positioning: By placing the sling under the leg and holding the handles it is possible for a carer to safely reposition a person's legs in bed.



2. SAFETY

The NRS Handling Sling should not be used for lifting a person, or their legs, it is designed only as a repositioning and support aid.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. CLEANING INSTRUCTIONS.

- Machine wash warm (up to 40°C).
- Do not use bleach or biological detergents.
- Do not spot clean with strong chemicals.
- Do not autoclave.
- Do not dry clean.
- Tumble dry Warm. Reduced spin, or air dry at very low temperatures.

5. SPECIFICATION

Main body fabric and fleece: 100% polyester.

Webbing: 100% polypropylene.

Foam Inserts: 100% polyethylene.

6. DIMENSIONS:

Padding: 610 mm x 220 mm (24 x 8.5").

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from fabric and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address below.

If you have experienced an injury, contact us but feel free to contact the Medical and Healthcare products Regulatory Agency at:

devices.regulatory@mhra.gov.uk



NRS HEALTHCARE Sherwood House,
Cartwright Way, Forest Business Park, Bardon
Hill, Coalville, Leicestershire, LE67 1UB, UK.



NRS HEALTHCARE 4C Altona Road, Blaris
Industrial Estate, Lisburn, Co Antrim,
Northern Ireland BT27 5QB

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk