

User Instructions

P18808 Knee Pad Covers

GENERAL SAFETY WARNING

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.



1.0 INTENDED USE

The transfer aid knee pad covers offer additional comfort to the user of the Atlas Advance Transfer Aid. The extra cushioning provides greater protection for users with frail or thin skin, helping to increase pressure relief.

2.0 SAFETY

- Ensure the covers are correctly fitted over the knee pads prior to use.

3.0 INSTALLATION

Remove all packaging and check the knee pad covers carefully. If you suspect any damage, do not use and contact your supplier immediately.

- Position the knee pad cover over the knee pad such that the seam is on the outside edge, figure 1.
- Pull the top elastic strap over the top of the knee pad, figure 2.
- Stretch the cover to fold over the top of the knee pad, figure 3.

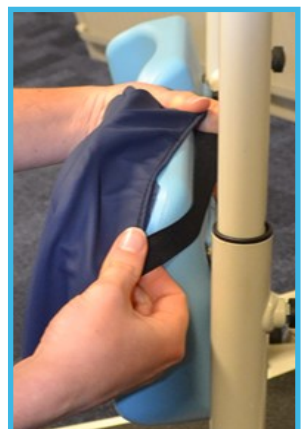


Fig. 1



Fig. 2

Fig. 3



- Pull the bottom elastic strap up around the bottom of the knee pad, figure 4.
- Stretch the cover to fit tightly over the bottom of the knee pad, figure 5.
- Repeat for the second knee pad and cover.



Fig. 4

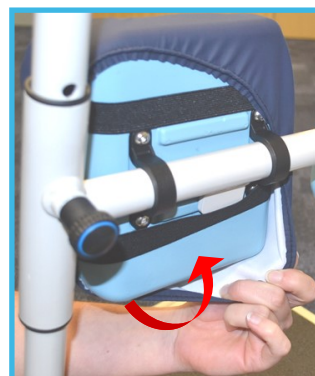


Fig. 5



Fig. 6 'Front View'



Fig. 7 'Back View'

4.0 CLEANING

- Remove the covers from the knee pads prior to washing. For spot cleans use a damp cloth.
- Machine wash at a maximum temperature of 40°C using a mild detergent. Do not use bleach. Do not steam clean. Do not iron.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the cover and invalidate any warranty.
- Allow covers to air dry thoroughly before refitting to knee pads. Do not tumble dry.

5.0 MAINTENANCE

- Periodically inspect the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

6.0 PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only. Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk