

User Instructions

Harrier Toilet Frame L84694

Maximum User Weight:



General Safety Warning

Please ensure these instructions are fully read and understood before installation/use. Retain these instructions in a safe place for future reference.

Intended Purpose

The Harrier toilet frame is designed to aid the user to sit and rise from the toilet. It has a low crossbar which enables it to fit fully under most toilets. The open rear design allows carers better access to the client.

Fitting Instructions

1. Turn toilet frame upside down on floor/ flat surface.
2. Adjust to the correct height by removing pin clips, sliding the extension legs to required position and replace pin clips.
3. Ensure all leg extensions are secured to the same adjustment hole to ensure even height, and prevent rocking.
4. Place the toilet frame over the toilet

Safety Precautions

- Do not exceed the maximum user weight limit.
- Ensure an even floor surface to prevent Rocking.
- Inspect at regular intervals for wear and damage.
- Do not use if rubber feet are damaged.



Cleaning Instructions

- Clean the frame and armrests regularly, use a general purpose, neutral detergent and warm water.
- Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.

Manufactured for
NRS Healthcare

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire.
LE67 1UB, UK

Tel. Main Office: 0330 100 00 98
Tel. Customer Services: 0845 120 4522
Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from NRS Healthcare. To view the full range visit www.nrs-uk.co.uk or request a catalogue