

The packaging of this mattress is sealed prior to despatch and the mattress itself will be clean prior to closure. If the mattress has become exposed or damaged during transit, please contact us at the address below.



USER MANUAL

(please read carefully)

COZY COMFORT MATTRESS

(M76230)

CMHR foam mattress. 2 way stretch, vapour permeable removable cover, sewn seams and zipped closure

NRS HEALTHCARE
SHERWOOD HOUSE, CARTWRIGHT WAY,
FOREST BUSINESS PARK, BARDON HILL,
COALVILLE LEICESTERSHIRE.
LE67 1UB

Tel. Main Office: 0330 100 00 98
Tel. Customer Services: 0845 120 4522
Email: customerservice@nrs-uk.co.uk

WWW.NRS-UK.CO.UK

Manufactured by Recticel Midlands
Azalea Close, Clover Nook Industrial Park, Alfreton, Derbyshire, DE55 4QX
Tel 01773 838000 Fax 01773 838870

Recticel Midlands is a division of Recticel Ltd
Registered Office: Bluebell Close, Clover Nook Industrial Park, Alfreton, Derbyshire DE55 4RA
Registered in England No. 665376



The manufacturing process operates under Class 1 of the Medical Devices Directive by fully complying with the essential requirements for CE marking this product.

Flammability

The Cozy Comfort mattress meets the requirements of BS7177:2008 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

User Instructions

The Cozy Comfort Mattress has been designed for use on solid / mesh, profiling / flat bed bases, by patients in the high risk user category[§]. with a weight guide of up to 171 Kgs / 27 stones.

Ensure that the surface of the bed is clear from debris before placing the mattress – debris can damage the coating of the fabric.

Frequent turning is strongly recommended for 4-way turning mattresses as all foam grades fatigue during use and turning will ensure loading is evenly spread across both sides and ends of the mattress.

The cover on this mattress is removable for cleaning purposes. Always ensure that, if removed, the cover is returned to the same inner. Using this cover on a different mattress core MAY render the warranty invalid.

The label on this mattress has a unique batch number which should be used in the event of a warranty claim. NEVER remove the label.

Instructions for use are screen printed onto the cover of the product for ease of reference.

[§] Indication of use based upon interface pressure readings– not a prescriptive criteria. Always defer to clinician's judgement.

Cleaning

- For general cleaning and superficial dirt, use a single use wipe with a solution of warm water and neutral detergent. Do not use abrasive or phenolic-based cleaners.
- To disinfect in situ, use a 0.1% sodium hypochloride solution (1,000ppm chlorine) If required, a 1% Chlorine solution (10,000ppm) and cold water may be applied for not more than 2 minutes
- Thoroughly rinse the cover with clean water and allow to dry before re-use or storage
- Removable, contaminated covers should be laundered at 65°C for not less than 10 minutes or at 95°C, for not less than 3 minutes. Do not mangle or iron.
- Spin & tumble dry at no more than 60°C

Warranty

Covered against faulty manufacture and materials as replacements goods.

Normal wear and tear expected would not be included.

All warranties subject to terms and conditions of trading.

Please refer to our warranty schedule for terms.

In order to claim under warranty refer to the batch number printed on the product label.

Storage & Care

- Store the mattress flat – do not fold
- Do not place heavy or sharp objects on the mattress surface when not in use
- Do not allow sharp objects to penetrate the cover coating
- Keep the mattress clean and use a protective cover when not in use
- Regularly inspect the interior and exterior of the mattress
- Do not store in damp conditions or next to radiators or other heating devices
- Store in a cool, dry area. Do not store directly onto the floor
- Do not drag or pull a mattress by its cover
- Adult mattresses should be handled by two people
- Mattresses should be transported on a bed or trolley suitable for the purpose