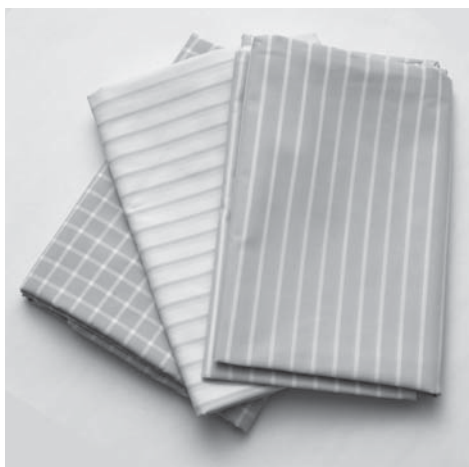
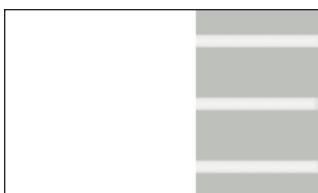
Design of fabrics, Mönster på tyget/stoffet

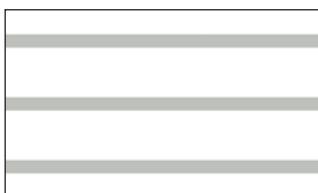
WendyLett
SystemRoMedic

Sliding sheet: PES sliding satin in the center and PES/Cotton on both sides.

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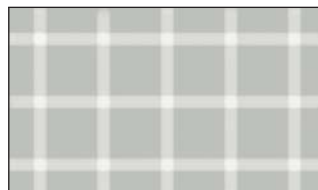
Visual inspection

Check that all materials are perfect and free from damage. Check whether the material has been subject to any wear and tear or has become faded.

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EN

Positioning

Position the WendyLett with the smooth surface in the centre of the bed and the draw sheet on top. After turning the patient you can prevent further sliding by tucking the draw sheet under the mattress on both sides of the bed.

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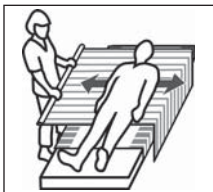
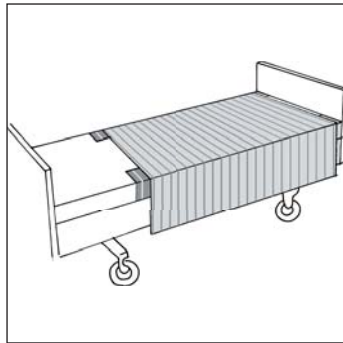
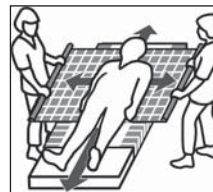
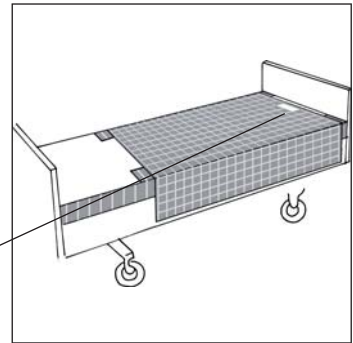
Grips and techniques.

Perform the turning by taking a firm grip on the draw sheet close to the patient's hip and shoulder. Fill your hand with fabrics for good ergonomic grips. Use straight arms, lean backwards, pull and lift.

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Manual

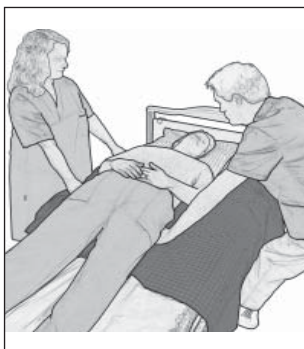
WendyLett2WAY SystemRoMedic

Turning a patient in bed with a draw sheet can be performed by two assistants. It is a cooperation where one assistant pulls in the draw sheet while the other pushes gently down and forward. After the turning you can prevent further sliding by tucking the draw sheet under the mattress on both sides of the bed.

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WendyLett4WAY SystemRoMedic

The draw sheet WendyLett4Way with the sliding material on the under part, is specially designed for turning heavy patients. The techniques are the same as described above for WendyLett2Way. After the turning you can prevent further sliding by tucking the draw sheet under the mattress on both sides of the bed. Before fixing the draw sheet under the mattress, you should even out any folds under the patient. Take a grip on both sides of the draw sheet and pull diagonally. Do not leave the patient without putting up the bed rails.

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When pulling a patient higher up in bed, you take a firm grip in the draw sheet close to the shoulders on both sides of the patient. Stand in a walking position and take one step backwards. You can prevent the patient from sliding down again by folding a part of the draw sheet in such a way that it is double folded under the hips.

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WendyLett4WAY
SystemRoMedic

Manual no: 00846 Ver. No 4 071005
EAN no 73 31769 00389 0

You can prevent sliding down in bed even in a sitting position by using the WendyLett4Way. Fold a part of the draw sheet in such a way that it is double folded under the hips. Tuck the draw sheet under the mattress but only the part close to the hips.

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The WendyLett4Way is very useful when you are going to set a patient up on the edge of the bed. Raise the head end of the bed. Take a good grip on the draw sheet just below the shoulder part on each side of the patient. One person should assist with the legs when you turn the patient around.

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Note:

Do not leave the patient without putting up the bed rails.

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Information:

For further information, contact your specialist in transfer, our representative or www.romedic.com

Ensure that you always have the latest version of the manual. It can be downloaded from our website: <http://www.romedic.com/manuals.htm>

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RoMedic AB