

# User Instructions

## M48465 Economy Drop Down Rail with Leg

### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### Maximum User Weight



### 1.0 Intended Use

The Economy Drop Down Rail with Leg is a wall mounted rail which provides a stable support to users when lowering onto and raising from the toilet. It can be pushed back against the wall to enable side transfers on and off the toilet seat or when not in use. It is not designed for use in a shower.

### 2.0 Safety Precautions

Do not exceed the maximum user weight of the product 21 st, 136 kg.

Do not attempt to climb on or swing from the product.

### 3.0 Unpacking

Remove packaging and check product carefully. If any parts are damaged, do not use and contact your supplier immediately.

### 4.0 Installation Precautions

This product must be installed by a suitably qualified and competent person.

Always check the mounting surface and substrate are structurally sound, strengthening modifications may be required.

Always check for concealed electricity cables and water pipes prior to installation.

**Please Note:** As the mounting substrate can vary considerably, we do not supply fixings. Please ensure the chosen fixings are fit for purpose and are compatible with both the product and the mounting substrate. If in doubt, please contact a suitably qualified person.



Figure 1 – M48465 In Use

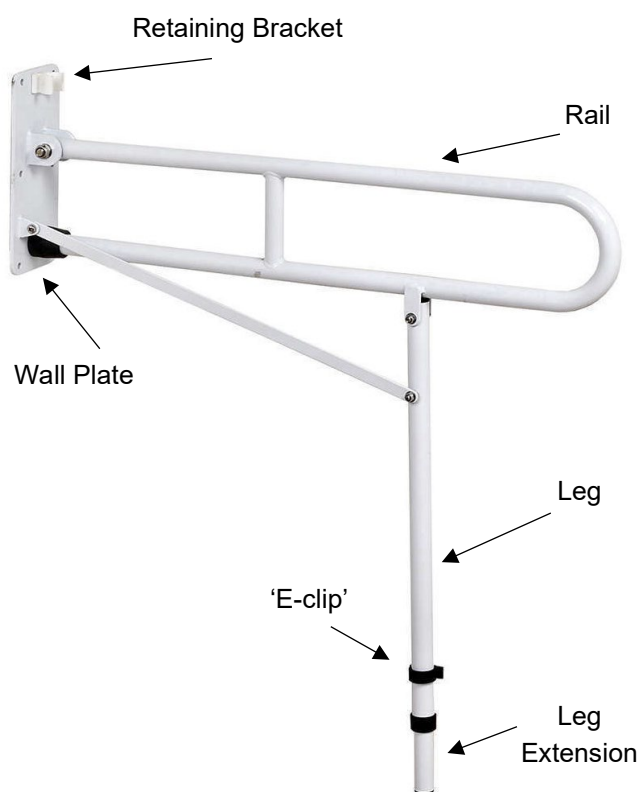


Figure 2 – Parts List

## 4.1 Instructions

Position the rail such that it provides comfortable support to the user and can be lowered and raised without interfering with any other furniture or fixings.

Extend the leg to the desired height by removing the 'E-clip' and sliding the inner tube to the required position.

Align the holes and reinsert the 'E-clip'.

Position the wall plate flat against the wall with the leg square and in contact with the floor.

Mark the position of the 6 holes.

Drill the 6 holes sized appropriately for the chosen fixings.

Locate the wall plate over the holes and secure in place.

Check the rail is secure and functions correctly without interfering with any other furniture or fixings.

Tighten and inspect all fixings prior to use.

## 5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Always dry the product fully after cleaning.

## 6.0 Maintenance

Check the rail operates smoothly and is securely held by the retaining bracket when pushed up against the wall.

Check the 'E-clip' is fitted correctly.

Periodically spray a water dispersing agent (e.g. WD40) on the hinged joints.

Regularly check the product is rigid and stable and that all wall mounted fixings are secure and tight.

Inspect regularly for signs of wear or damage. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

## 7.0 Product Disposal

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

## 8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

## NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

Tel. Customer Services: +44 (0)345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

**NRS Healthcare is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completcareshop.co.uk](http://www.completcareshop.co.uk)**