

BATTERY CARE INSTRUCTIONS

Recharge battery between uses wherever possible, as frequent charging will maximise overall battery life. It is recommended that you leave the battery on charge for a minimum of 2 hours.

Only use the approved Relaxa charger to recharge the battery pack.


The battery pack should be removed from the Relaxa unit when not in use for more than a few days.

Do not allow the battery pack to become wet.

Do not dismantle the battery pack or attempt to poke foreign objects into either connector.

When not in use the battery should ideally be stored between 20-25°C.

A battery that is not in use will self-discharge over long periods, it should be recharged at least once every six months.

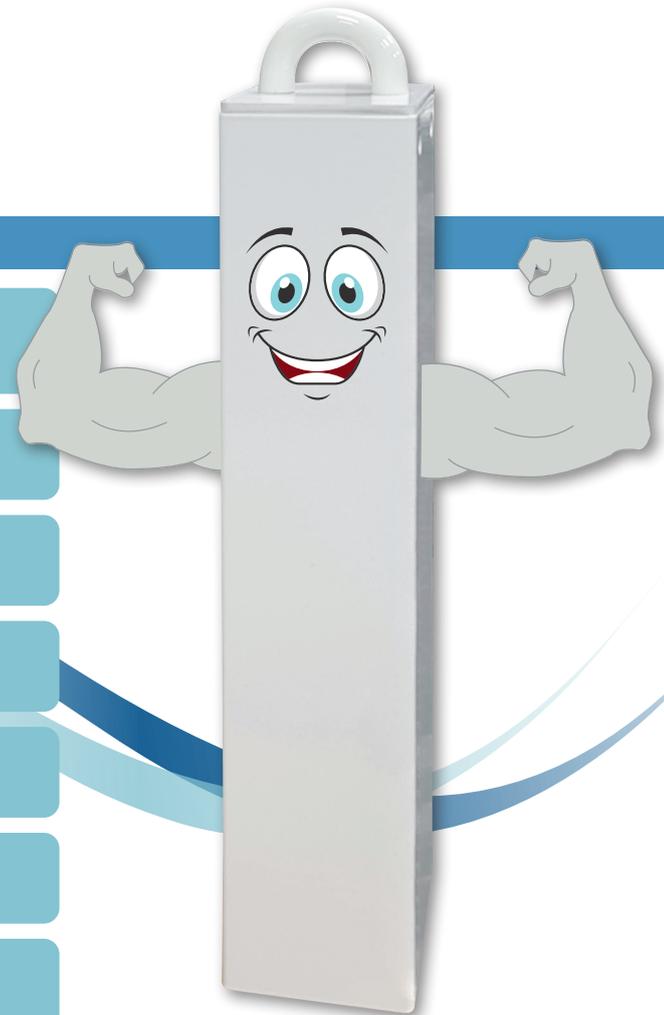

INSTRUCTIONS POUR L'ENTRETIEN DE LA BATTERIE

Rechargez la batterie entre chaque usage, si possible.
Recharger fréquemment permettra de maximiser la vie de la batterie. Il est recommandé de recharger la batterie durant deux heures minimum.

Utilisez uniquement le chargeur original Relaxa pour recharger la batterie.

Retirez la batterie de l'unité Relaxa lorsque vous ne l'utilisez pas pendant plusieurs jours.

Laissez la batterie hors de l'humidité.

N'essayez en aucun cas de démonter la batterie ou d'introduire des objets dans les connecteurs.

Lorsque vous ne l'utilisez pas, la batterie doit être conservée entre 20-25°C.

Lorsque la batterie n'est pas utilisée pendant une longue période, elle s'autodécharge. Pensez à recharger la batterie au moins une fois tous les six mois.

