



THE LOCOMOTOR RANGE

LOCOMOTOR S.U.P.A. SLIDE



INSTRUCTIONS FOR USE

LOCO-200 LOCOMOTOR S.U.P.A. Slide Standard 75cm x 200cm Box 100

LOCO-201 LOCOMOTOR S.U.P.A. Slide Wide 100cm x 200cm Box 100

<https://selecthealthcare.co.uk/13-sup-a-slide>

IMPORTANT:

PLEASE RETAIN THIS USER INFORMATION SHEET WITH THE PRODUCT AND REFER TO IT BEFORE USE.

DESCRIPTION & PURPOSE:


The LOCOMOTOR S.U.P.A. Slide is a disposable / single patient use slide sheet available in standard and wide sizes.

The S.U.P.A. Slide can be used either in pairs or singularly to reposition or transfer a patient in conjunction with a bed, trolley, theatre table, examination couch or similar. S.U.P.A. Slides are delivered in a dispensing box of 100 sheets.

INSTRUCTIONS FOR USE:


Open the dispensing box as per instructions printed on the box and remove the required number of sheets.

USED IN PAIRS

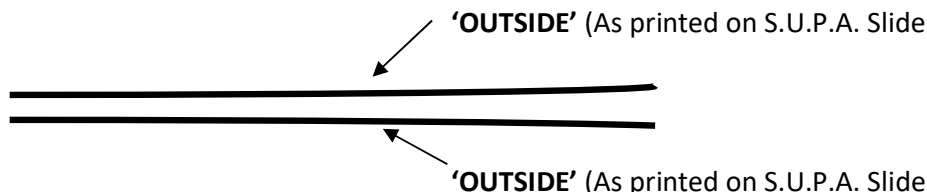

- Place one S.U.P.A Slide on top of another ensuring that the repeatedly printed instruction 'OUTSIDE' is on the outside of each S.U.P.A. Slide.
- Using an approved manual handling technique roll the patient and insert the sheets underneath the patient. Ensure that any part of the patient that is not assisting with the transfer is on two layers of S.U.P.A. Slide.
- To perform a transfer, take hold of the top S.U.P.A. Slide and slide the patient to the required position.
- The S.U.P.A. Slide can then be removed by folding a corner of each sheet back underneath itself and sliding one layer against the other until free of the patient. Steady the patient as the S.U.P.A. Slide is removed.

USED SINGULARLY

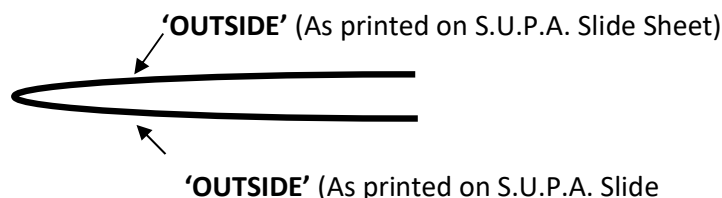

- Fold the S.U.P.A. Slide to form a loop (open tube), ensuring that the printed instruction 'OUTSIDE' is on the outside.
- Roll the patient and insert in the desired position. Ensure that any part of the patient that is not assisting with the transfer is on two layers of S.U.P.A. Slide.
- To perform a transfer take hold of the top layer of the S.U.P.A. Slide and slide the patient to the required position. The S.U.P.A. Slide can then be removed by folding a corner of the sheet back underneath itself and sliding one layer against the other until free of the patient. Steady that patient as the S.U.P.A. Slide is removed.

SINGLE PATIENT USE ONLY, DO NOT LAUNDER. DISCARD IF SOILED OR DAMAGED

For information about sliding techniques contact your Moving and Handling Trainer, Back Care Advisor or equivalent.

All patient moving and handling tasks should be carried out in accordance with approved patient moving and handling techniques and after a full assessment of the patient, equipment, task and environment.

Always follow the approved policy of your place of work when adopting moving and handling practices.

INSPECTION:

Before each use the S.U.P.A. Slide must be inspected for damage or soiling. **Do not use** if the S.U.P.A. Slide is showing any signs of damage or soiling.

STORAGE:

Store in dry conditions away from direct sunlight and sources of heat.

MATERIAL:

Spun bonded, non-woven, silicone coated polypropylene.

DISPOSAL:

Please refer to the policy of your place of work for the disposal of non-hazardous waste.

For Technical help or for more information

please contact your supplier or contact customer services at:



Select... Health Care (U.K) Ltd. Tel: (01933) 411729
Unit 2, Albert Rd Fax: (01933) 410213
Rushden mail: info@selecthealthcare.co.uk
Northants web Site: www.selecthealthcare.co.uk
NN10 0BU product data:
<https://selecthealthcare.co.uk/13-supra-slide>