

User Instructions

Extra Wide Commode Range

N10699 Fixed Arms - 550mm wide

N10705 Fixed Arms - 660mm wide

N10717 Removable Arms - 660mm wide

**Maximum
User Weight**



General Safety Warning

Please ensure these instructions are fully read and followed. Failure to do so may result in injury to the user. Retain these instructions in a safe place for future reference.

Intended Use

The Extra Wide Commode Range is primarily intended to be used as a commode for toileting. It may also be used as a seat support over a standard toilet (with backrest and existing toilet seat removed) and, when the seat cover is fitted, as a short duration bedroom seat/chair or for occasional use in a shower. Side transfers are possible with N10717 - removable arm version. The seat height is adjustable on all variants.

Unpacking

Remove all parts from packaging, Check all parts are present, refer to contents list below. Check for damage. If you see, or suspect a fault, do not use the product and contact your supplier.

Contents List

Item	Description	N10699	N10705	N10717
1	Commode Frame	1	1	1
2	Toilet Seat	1	1	1
3	PU Seat Cover	1	1	1
4	Potty	1	1	1
5	Potty Lid	1	1	1
6	Backrest	1	1	1
7	Leg	4	2	2
8	Leg/wheel Assy LH	-	1	1
9	Leg/wheel Assy RH	-	1	1
10	Removable Arm (not shown)	-	-	2
11	Seat Fixing Screw (not shown)	2	2	2
12	Seat Fixing Nut (not shown)	2	2	2
13	Washer (not shown)	2	2	2
14	E- Clip	4	4	8



Assembly Instructions

- With the frame upturned, fit the legs, at the required height, using the e-clips provided.
- For N10699 fit all four legs. For N10705 & N10717 fit the two legs without wheels to one side of the commode frame; then fit the legs with wheels to the other side of the frame with the wheels in the orientation as shown in the picture above. Turn the commode onto its legs and check the legs are at the same height and the frame does not rock.
- To fit the toilet seat first unclip the plastic cover on the seat hinges to reveal the fixing holes
- Align the seat fixing holes with the holes in the frame.
- Place a washer on each fixing screw, and insert them through the holes; secure from the other side with the nuts provided. Note: the seat must be secure, but do not over tighten the nuts.
- Clip the plastic covers back to the hinge, to cover the screws.
- Fit the backrest in the desired position. Tip: the backrest slides more easily into position if both tubes are pushed into the frame at the same time until the button clip engages to secure in place. The backrest has three seat depth settings, to move further inwards, depress the spring button clips and push the backrest further into the frame.
- Slide the potty into the commode, from the front, using the rail wires on the underside of the frame
- Place the potty lid on the potty if required
- Fit the PU seat cover over the toilet seat.

N10717 only

- Fit the removable arms, using the e-clips provided, as shown in the picture above.

Safety Precautions

- Ensure an even floor surface to avoid rocking. All ferrules must be in contact with the floor at all times.
- Do not exceed the maximum user weight limit.
- It is recommended a risk assessment is carried out by a healthcare professional with regard to the environment and capabilities of the user.
- Inspect at regular intervals for damage or excessive wear, if found, or suspected, remove from use immediately and contact your supplier.
- Always dispose of the potty contents into a toilet or sluice.

Instructions for use

- For N10705 and N10717 only: to manoeuvre into position, tilt onto the wheels by lifting the armrest and tilting the commode upwards so the wheels are in contact with the floor and wheel into position.
- Set the product to the correct height for the client by removing the e-clips which secure the legs and slide them to the required height, align the holes in the leg and frame and secure with the e-clips. Ensure all legs are at the same height.
- If the product is to be used over a toilet, as shown below, remove the backrest and ensure the height is set so all parts of the commode clear the toilet. You may need to remove the existing toilet seat also. Ensure the aperture of the commode seat is positioned centrally over the toilet bowl.
- N10717 Removable arm version: to remove the arm, unclip the two e-clips which secure the arm and pull gently upwards. When replacing ensure both e-clips are re-fitted, and the arm is stable.
- When using the potty, slide fully into position from the front. To help contain odours fit the potty lid when transporting.
- Always dispose of the potty contents into a toilet or sluice.



Toilet Fitting

Product Care: Cleaning General

- To help reduce the risk of infection as well as keeping the commode clean and odour free, it is recommended a sanitising spray or cloth is used to wipe down the potty, commode seat and arms after every use.
- Carers should always wear protective gloves when carrying out these tasks.

Product Care: Cleaning the Commode Potty

- After every use rinse the potty thoroughly with warm water, ensure there is no residue, clean with a mild disinfectant.
- Use of sanitising sprays or cloths is recommended

Product Care: Cleaning the Commode Frame

- Clean the frame regularly, at least on a weekly basis, with a mild disinfectant solution.
- This product is also suitable to be cleaned in commercial high temperature washing machines.
- Ensure all parts are dried thoroughly before re-use.

Product Care: Maintenance

- Inspect the product at regular intervals for signs of wear, damage or possible faults. If any are found or are suspected, remove from use immediately and contact your supplier for advice.

The Extra Wide Commode Range is part of a range of over 3000 daily living aids available from NRS Healthcare. To view all of our products please visit our website: **www.nrs-uk.co.uk** or request a catalogue from the number below.

NRS Healthcare

Sherwood House, Cartwright Way
Forest Business Park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB, UK

Tel: 0845 120 4522

Email: customerservice@nrs-uk.co.uk

www.nrs-uk.co.uk

