

## Product Technical Data Sheet 005

### Dovedale Commode Range

#### Product Range

F18864	Dovedale Commode Adjustable Height
F19455	Dovedale Commode Adjustable Height with Detachable Arms
M11181	Dovedale Commode Extra Low—Adjustable Height

#### Product Description & Intended Purpose

The Dovedale Commode is designed to assist those who have difficulty in accessing or using a standard toilet.

#### Product Features

- Height adjustable and stackable.
- Powder coated metal frame.
- Hinged plastic toilet seat over a commode pan with lid.
- Padded vinyl lift off vanity cover.
- Padded fixed back.
- Removable Arm Version
- Extra low seat height version

#### Technical Specification

- Seat Height: 450mm min—600 max
- Width between arms, top: 510mm
- Width between arms, bottom: 410mm
- Backrest height from seat: 190 & 355mm
- Armrest Height from seat: 200mm
- Toilet seat Dimensions: 380 x 360mm
- Footprint width & depth max: 495W x 495D
- Footprint width & depth min: 485W x 455D
- **Maximum User Weight:** 28st/180kg
- **Product Weight:** 6.6kg
- **Pack Type:** Product is delivered stacked 44 per pallet

#### Installation Information *(Note: for detailed information refer to User Instructions)*

- Product is delivered assembled (the padded vinyl cover rests on the toilet seat only—it is not fixed).
- Seat height is adjusted by the leg extensions (refer to UI for detailed instruction).

#### Safety Precautions

- Avoid extending the legs in a manner which causes the commode to slant.
- Do not move the commode whilst occupied.
- Do not use an open sink or drain when emptying the commode. Contents should be disposed of in a proper sluice or WC.

## PTDS005

### Dovedale Commode Range

#### Cleaning

- To clean the vinyl seat pads use a suitable detergent and warm water, wipe seats and dry with a disposable paper towel.
- To clean the steel frame and toilet seat use a suitable detergent and warm water. Wash framework thoroughly, particularly the toilet seat hinges and underside of lid.
- DO NOT use abrasive cleaners or cloths on any part of this product.
- All parts apart from the vinyl vanity cover can be steam cleaned.
- All parts apart from the vinyl vanity cover can be autoclaved at 80°C if required.
- Always allow product to fully dry prior to stacking and storage.

#### Care & Maintenance

- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the feet/ferrules.
- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Check for general wear or damage. Any damage should immediately be reported to the supplier and the product taken out of action.

#### Accessories / Related Products

- L99351 Disposable Care Bag Commode Liners— Biodegradable
- M66364 Disposable Care Bag Commode Liners—Non Biodegradable
- M11193 Commode Potty & Lid

#### FAQ's

xxx

#### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK