

USER INSTRUCTIONS

HEIGHT ADJUSTABLE SHOWER STOOL

P01560

NHeight Adjustable Shower Chair

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

For use in a bathroom or shower, to provide a stable seated surface for clients with reduced mobility who are unable to stand during showering.



2. SAFETY

Warning: The maximum user weight must not be more than 160 kg (25 st).

Do not use if the feet are damaged.

Ensure all legs are secured at the same height to keep the chair stable.

Do not attempt to move the chair when occupied.

3. UNPACKING

This product is supplied fully assembled.

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try to use.

4. HEIGHT ADJUSTMENT

1. Remove 'E-clip' and slide leg extension to required height.
2. Align the holes and re-insert the 'E-clip'.
3. Repeat for all four legs ensuring they are set to equal height.
4. Check the seat is level and the product is stable before use.

5. MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Check that all fixing clips are inserted correctly and there is no excessive movement in the legs or seat.

If you see or suspect a fault do not use and contact your supplier immediately.

6. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

Do not autoclave.

Allow product to fully dry prior to use/storage.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.	
Tel: Customer Services: +44 (0)345 121 8111	Email: customerservice@nrshealthcare.co.uk
NRS Healthcare is a leading supplier of community equipment and daily living aids. Please visit one of our websites to view our full range of products: www.healthcarepro.co.uk or www.completecareshop.co.uk	