

L97718

Economy Shower Stool

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

Intended to be used in a bathroom or shower, to provide a stable seated surface for users with reduced mobility who are unable to stand during showering.



2. SAFETY

Warning: The maximum user weight must not be more than 136Kg (21st).

Inspect at regular intervals for wear and damage.

Do not use if rubber feet are damaged.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents

1. Bottom Leg 1 x 1
2. Bottom Leg 2 x 1
3. Long Screw x 5
4. Cupped Washer x 5
5. Seat x 1

4. ASSEMBLY/INSTALLATION

Tools required: Cross Slot Screwdriver

Maximum User Weight



1. Place seat upside down on floor/flat surface
2. Align seat legs on the seat according to the numbers indicated on the legs and seat.
3. Install mounting screws through the cupped washer, top leg and bottom leg and tighten to the seat. Check seat is secure and stable.
4. Adjust to the correct height by depressing the snap buttons and sliding the extension legs up or down the frame tube until it snaps into the appropriate adjustment hole.
5. Ensure all four leg extensions are secured to the same adjustment position to ensure even height. Ensure all snap buttons are fully visible and protruding.

5. IN USE

Place in shower. Ensure all four legs are level and in contact with the shower floor.

Sit centrally on the shower stool.

6. CLEANING INSTRUCTIONS.

Clean regularly, wiping down with mild detergent / disinfectant solution.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

www.healthcarepro.co.uk or www.completecareshop.co.uk