

USER INSTRUCTIONS

KEY TURNER RANGE

M10179	Single Key Turner	M10188	Double Key Turner
G45769	Triple Key Turner		

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The NRS Healthcare Key Turner Range enables people with reduced hand function or weak grip to turn a door key more easily.



2. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3. USER INSTRUCTIONS

1. Locate the white handle at the side of the product.
2. Turn anti-clockwise until it separates from the key turner.
3. Remove the brass screw.
4. Insert key into one of the spaces, lining up the hole in the key with the screw hole.
5. Insert screw through the key turner and key.
6. Replace white handle to threaded end of the screw and tighten in clockwise direction.

Note: The key turner will only take small Yale type keys, not thicker Chubb type keys.

Note: The key will need a minimum 5mm hole for the screw to fit through.

4. PRODUCT DISPOSAL

This product is manufactured from metal and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

5. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk