

The *butterfly* rampTM

Instructions for use

Product codes: DVBM, DVBL



Before use



- Remove and retain all packaging.
- Ensure the ramp has not been damaged in transit or storage.
- Carefully read the instructions and warning texts before using ramp. Keep instructions for reference.
- Ensure the surface at both levels is firm, stable and able to bear the intended load.

To Deploy

1. To carry the ramp, use special finger-safe carrying hole.



2. Position the ramp to bridge over the threshold. The Butterfly Ramp spine must sit firmly flat and level on the threshold.

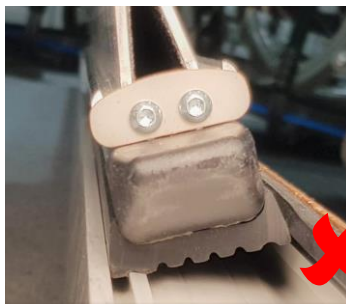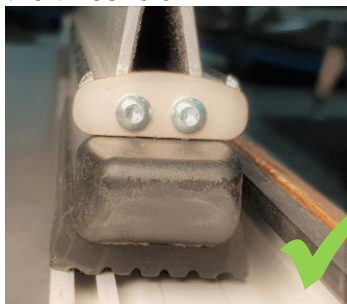

3. The Gripline ends of both sides of the ramp must rest firmly on the level surface.



4. The Butterfly ramp can safely span the lock keep for double doors



5. Ensure the ramps do not become dislodged in use. Check ramps are firmly located, and stable each time they are used. Take care when using the ramps, so that no part of the wheelchair catches and moves the ramps.

RampCentre is a registered trademark of Enable Access, AL9 5SD. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 09 Dec 2020



Enable Access, Marshmoor Works, Great North Road, North Mymms, Hatfield, AL9 5SD, UK.



Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

For technical support or details on your nearest stockist please call 020 8275 0375



The *butterfly* ramp™

Instructions for use

Product codes: DVBM, DVBL

Maintenance

- If the ramp becomes damaged or slippery in any way it should not be used (contact supplier/manufacturer).
- The ramp may be cleaned with normal cleaning products and left to dry before use.
- Maximum recommended cleaning temperature 50°C; please enquire for further details on use with industrial cleaning machines.

Recycling

- If you no longer require this product and it has been supplied to you as loan equipment from your local authority or a charity, please contact the service who loaned it to you and discuss its return/removal from your property. If this is your own product and you wish to dispose of it, please liaise with a local recycling service.
- All the materials in the ramps are 100% recyclable. Detach all rubber detail and send the aluminium parts to metal recycling.

Warranty

- The ramp carries a comprehensive 2-year warranty for normal use, conditions apply.
- Use other than published/intended will void the warranty.
- The ramps are not designed for permanent outside use and must not be fixed down.
- Any modification/repair other than by manufacturer will void the warranty.



Key Product information

Code	Height Range		Width	Overall Length	Capacity	Weight	Stored Dimensions L x H x D
	From	To					
DVBM	Zero	10cm plus (4ins plus)	70cm (28ins)	90cm (36ins)	350kg (55st)	6.8kg (15lbs)	70 x 48 x 4cm (28 x 19 x 1.5ins)
DVBL	Zero	15cm plus (6ins plus)	70cm (28ins)	120cm (48ins)	300kg (47st)	9.1kg (20lbs)	70 x 63 x 4cm (28 x 25 x 1.5ins)

RampCentre is a registered trademark of Enable Access, AL9 5SD. We undertake a policy of continual review and research, which may lead to changes in product appearance and specification. Date of Issue: 09 Dec 2020



Enable Access, Marshmoor Works, Great North Road, North Mymms, Hatfield, AL9 5SD, UK.



Enable Access Global Ltd, 27 Phibsboro Place, Dublin 7, D07 V20, Republic of Ireland

For technical support or details on your nearest stockist please call 020 8275 0375