

F19571	Single Handle	L33194	Double Handle
--------	---------------	--------	---------------

### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

### 1.0 Intended Use

The leg lifter can offer greater mobility and independence to the user providing support for lifting stiff or immobile legs. Particularly useful when getting in and out of bed, a wheelchair or car but also for repositioning legs when sitting.

### 2.0 Safety

Do not leave product on floor, it could become a trip hazard.

Take care when using the lifter not to get tangled. Do not loop the handle over the wrist.

### 3.0 Unpacking

Remove packaging and check product carefully. If you suspect any faults, do not use and contact your supplier immediately.

### 4.0 Instructions for Use

Ensure the foot loop is open wide, mould it to roughly the correct shape and size of your foot.

Hold the handle (F19571) or handles (L33194) and use them to help you hook the foot loop over your foot.

Once your foot is comfortably inside the loop, use the handles to lift your leg up and guide it to the new position.

Once your leg is in the desired position remove from your foot, repeat the process if necessary for your other leg.



Figure 1 - F19571 Single Loop



Figure 2 - Using the Single Loop Leg Lifter

## 5.0 Cleaning

Clean regularly using a general purpose, neutral detergent and warm water. Do not use bleach.

Hand wash at a low temperature using a mild detergent. Do not machine wash.

Allow product to fully dry prior to use/storage. Do not tumble dry.

## 6.0 Maintenance

Periodically inspect the product for signs of wear or damage, pay particular attention to the seams and stitching. If you suspect any faults do not use and contact your supplier immediately.

## 7.0 Product Disposal & Waste

Thoroughly clean product before disposal.

Product is manufactured from plastic and textiles and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

## 8.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

### NRS Healthcare

Sherwood House, Cartwright Way,  
Forest Business park, Bardon Hill,  
Coalville, Leicestershire, LE67 1UB, UK

Tel. Customer Services: +44 (0)345 121 8111

Email: [customerservices@nrshealthcare.co.uk](mailto:customerservices@nrshealthcare.co.uk)

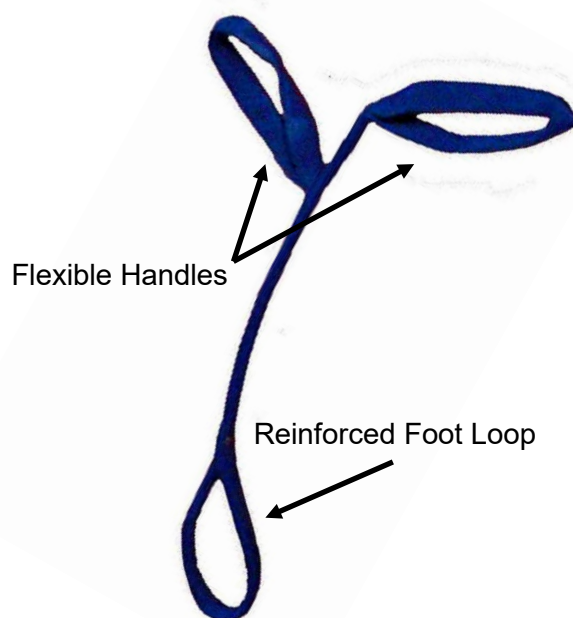


Figure 3 – L33194 Double Loop Leg Lifter



Figure 4 – Using the Double Loop Leg Lifter

**NRS Healthcare is a leading supplier of community equipment and daily living aids.**  
**Please visit one of our websites to view our full range of products:**  
**[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)**