



User Instruction for BA-7242 (Bed Pan)

Please read and retain this instruction, it provides essential information for correct and safe use.

This instruction is guideline only.

INTRODUCTION

The bed pan is a device that allows a person in bed to urinate or have a bowel movement. It is made of plastic with a handle on the back of the bed pan for cleaning.

SPECIFICATION

Overall Size	Weight	Material	Max. Load Weight
390 x 265 x 45 mm	405 g	PE	150 kgs

WARNING

- Manual cleaning safe up to 50°C.
- The max. load weight must under 150 kgs.
- Periodically inspect for scrape, wear and damage.

MAINTENANCE

The bed pan can be cleaned with a mild soap and a brush. The temperature must under 50°C.

Do not use solvents, such as acetone, lacquer thinner, lighter fluid or turpentine. If needed, clean the bad pan with a disinfectant soap or cleaning solution.

