

## User Instructions

### Shower Chair Range

F19054	Fixed Height	G31199	Fixed Height, Mobile
G31206	Height Adjustable	L64488	Height Adjustable, Mobile

#### GENERAL SAFETY WARNING

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

**Maximum  
User Weight**



#### 1.0 INTENDED USE

The shower chair is designed to provide stable seating for users with reduced mobility who are unable to stand whilst showering. The static models (F19054 & G31199) are suitable for use on a shower tray whilst the mobile models (G31199 & L64488) are ideal for wet rooms.

#### 2.0 SAFETY

- Always perform a risk assessment prior to use, including safe moving and handling guidelines.
- Do not exceed the maximum user weight; 31<sup>1</sup>/<sub>2</sub>st (200kg).
- Do not use if castors or ferrules are damaged.
- Do not attempt to climb or stand on the product.

##### **Models F19054 & G31199**

- Ensure all four legs are securely fixed at the same height prior to use.

##### **Models G31206 & L64488**

- Lock all four castors before transferring the user to and from the chair or when leaving the user.
- Plan the route to avoid obstacles prior to use. Only use on level surfaces and take care over door thresholds.

#### 3.0 UNPACKING

Remove all packaging and check parts carefully. If any parts are missing or appear damaged do not use and contact your supplier.



**G31206 Adjustable Height**



**L64488 Height Adjustable, Mobile**

#### 4.0 ASSEMBLY

- Product arrives fully assembled.

##### ***To Adjust the Height (G31206 & L64488)***

- Remove 'e-clip'.
- Slide the leg to the desired height and align the holes.
- Re-insert the 'e-clip', check the pin is fully located.

*Ensure all four legs are secured at the same height to prevent rocking or instability.*

#### 5.0 CLEANING

- Clean regularly using a general purpose neutral detergent and warm water.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- The shower chair is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 80°C.
- Always allow product to fully dry prior to use/storage.

#### 6.0 MAINTENANCE

- Check that all fixing clips are inserted correctly and there is no excessive movement.
- **F19054 & G31199** - Check the condition of the ferrules.
- **G31206 & L64488** - Check the castors are securely fitted and are free to rotate. Remove any build-up of dirt or debris.
- Periodically inspect the product for signs of wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

#### 7.0 PRODUCT DISPOSAL

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

#### 8.0 PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.



**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK

**NRS Healthcare** is a Leading Supplier of Community Equipment and Daily Living Aids, to view our full range, visit our website:

**[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk)**

UI/F19054/01/18/Iss5