



## PRODUCT INSTRUCTION SHEET



REFERENCE 190 Belgrave Bariatric Commode (Nov 13)

### DESCRIPTION

All welded steel frame with cellular moulded (anti-bacterial) backrest and over seat, with clip/on-off rota moulded seat and circular 14L potty.

### TECHNICAL SPECIFICATIONS

Adj Seat Height:	17" – 23"
Width between arms:	24"
Seat Depth:	17"
Seat to top of arms:	180mm
Seat to top of back:	420mm
Orifice width:	250mm
Orifice depth:	280mm
Product weight	13kg



### USAGE REQUIREMENTS

1. Use on level surfaces only, do not exceed user weight of 280kg (44 stone)
2. Where the height adjustment is required, ensure that there is no gap between horseshoe clip and frame.
3. As 2. above, the clip spindles also apply to the side arms.
4. Ensure users do not “swing-on” the frames either in a forward or backward motion. All four legs of the unit must be adjusted to the same height, and be in contact with the floor at all times.
5. The chair is designed to withstand temperatures provided by the latest disinfecting commercial washing machines. Otherwise, use a soft cloth dipped in neutral detergent and warm water. Do not use harsh abrasive materials. To disinfect, clean as above and wipe over with a solution of one thousand parts per million chloride disinfectant-rinse and dry.