Bed Ladder

N94561

GENERAL SAFETY WARNING

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury. Retain these instructions in a safe place for future reference.

1.0 INTENDED USE

The Bed Ladder is for use by those who have difficulty rising to a seated position in bed. It enables the user to pull themselves upright to a sitting position by pulling on successive handles.

This product can be fitted to most bed types and sizes from single to king size.

2.0 SAFETY PRECAUTIONS

- Always fit the bed ladder on top of the bed and bedding; do not fit between sheets.
- Ensure the ladder is securely attached to the foot of the bed prior to use.
- Ensure any excess or loose strapping is tucked or tied out of the way at all times.
- Do not exceed the maximum user weight, 22st 140kg.
- Keep the product away from fire.
- Not suitable for use by children.

3.0 UNPACKING

Remove packaging and check all parts carefully. If any parts are missing or appear damaged do not use and contact your supplier.

Contents:

Bed Ladder X 1

Bed Straps with plastic buckle X 2

4.0 FITTING

1. Loop a bed strap around one of the bed legs or castors at the foot of the bed.
2. Thread the buckle through the loop in the strap and pull tight, figure 1.
3. Repeat steps 1 & 2 for the second strap on the opposite bed leg/castor.
4. Place the Bed Ladder on the bed in the required position for the user, handles facing upwards.

The bed ladder can be positioned centrally on a single bed, or offset to one side on a double bed.

The top handle should be within easy reach of the user when lying down in the bed.

5. Use the bedding clips at the top of the ladder to secure it to the bed linen, figure 2.

If the bed linen is thin material, it may be necessary to fold twice for the bedding clips to grip securely.

6. Fasten the bed straps to the ladder using the buckle clips, ensuring the straps are not twisted.
7. Pull the straps tight adjusting the length as required. Tuck any excess strap under the mattress, figure 3.
8. The ladder is now fully fitted and is ready for use, figure 4.

Figure 1

Figure 2

Figure 3

5.0 USE

To use the bed ladder, the user should grasp the first handle and then reach for successive handles to pull themselves from a lying to an upright position:

Figure 4

6.0 CLEANING

Spot clean using a general purpose, neutral detergent and warm water.

Hand wash using a neutral detergent at a low temperature, maximum 30°C.

To disinfect, wash using a mild disinfectant and rinse thoroughly with fresh water. Please check the suitability of the cleaning product before use. Do not use bleach.

Allow the product to dry thoroughly prior to use/storage. Do not tumble dry or iron.

7.0 MAINTENANCE

Check the stitching along the seams and fixing points for the ladder handles.

Check the fabric for tears, holes or loose threads.

Check the buckles are correctly fastened and the straps are secure around the bed feet or castors.

Periodically inspect for signs of general wear. Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

The recommended life of the product is 5 years from the date of purchase and 8 years from the date of manufacture. Do not use the product beyond these dates.

8.0 PRODUCT DISPOSAL

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from textiles and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9.0 PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship. This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare,

Sherwood House, Cartwright Way,
Forest Business Park, Bardon Hill,
Coalville, Leicestershire, LE67 1UB

Tel. Head Office: 0330 100 0098
Tel. Customer Services: 0345 121 8111
Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment
and Daily Living Aids, to view our full range, visit our website:

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