

# USER INSTRUCTIONS

## HARRIER TOILET FRAME

L84694

Harrier Toilet Frame

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

### 1. INTENDED USE

The Harrier toilet frame is designed to aid the user to sit and rise from the toilet. It has a low crossbar which enables it to fit fully under most toilets. The open rear design allows carers better access to the user.



### 2. SAFETY

**Warning:** The maximum user weight must not be more than 159Kg (25st).

Do not exceed the maximum user weight limit.  
Ensure an even floor surface to prevent Rocking.  
Inspect at regular intervals for wear and damage.  
Do not use if rubber feet are damaged.

### 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### Maximum User Weight



### 4. ASSEMBLY/INSTALLATION

1. Turn toilet frame upside down on floor/ flat surface.
2. Adjust to the correct height by removing pin clips, sliding the extension legs to required position and replace pin clips.
3. Ensure all leg extensions are secured to the same adjustment hole to ensure even height, and prevent rocking.
4. Place the toilet frame over the toilet.

### 5. CLEANING INSTRUCTIONS.

Clean the frame and armrests regularly, use a general purpose, neutral detergent and warm water. Do not use an abrasive detergent, cleaner or cloth to clean or dry this product.  
Allow product to fully dry prior to use/storage.

### 6. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

### 7. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product. Any guarantee issued by NRS Healthcare does not affect your statutory rights.

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)