

## Product Technical Data Sheet 009 M11090 Toilet Frame with Seat

### Product Range

N/A | N/A

### Product Description & Intended Purpose

This product is designed to assist those who have difficulty in lowering onto or rising from a standard toilet. Primarily for use over the toilet using the splash guard provided but also for use as a commode with the addition of a potty (supplied separately).

### Product Features

- Height adjustable frame to suit a full range of users.
- Comfortable toilet seat complete with hinged lid.
- Removable chute for easy cleaning and improved hygiene.
- Can be used as a commode giving the option to fit a potty.



### Technical Specification

#### Dimensions (mm)

- Seat Height: 390 — 540 (15½" - 21¼")
- Seat Size: 420 X 350 (16½" X 13¾")
- Arm Height: 630 — 780 (24¾" - 30¾")
- Cross Bar Height: 350 — 500 (13¾" X 19¾")
- Overall Width: 530 (21")
- Width Between Arms: 490 (19")
- Aperture: 250 X 205 (10" X 8")
- Footprint Minimum: 520 X 530 (20½" X 21")
- Footprint Maximum: 520 X 580 (20½" X 23")

**Maximum User Weight: 190Kg (30st)**

### Installation Information: *(Note: for detailed information refer to User Instructions)*

- To adjust height, remove 'e-clips', slide inner tube to desired position, align holes and re-insert 'e-clips'.
- Place the frame over the toilet seat with the splash guard central over the toilet bowl.
- Check the frame is stable prior to use.

## PTDS009

### M11090 Toilet Frame with Seat

#### Safety Precautions

- Ensure all four legs are set to the same height to prevent rocking and instability.
- Do not exceed the maximum user weight of 190kg (30 stones).
- Do not climb on the product.

#### Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water. Pay particular attention to the toilet seat hinges and underside of lid.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
- Allow product to dry thoroughly prior to use/storage.

#### Care & Maintenance

- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules and 'e-clips'.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

#### Accessories / Related Products

- M20203 — Commode Pan
- F11572 — Floor Fixing Kit

#### FAQ's

None

#### Product Supplied By:

**NRS Healthcare** Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK