

# User Instructions

## L96738 - Wide Shower Stool

## L96749 - Extra Wide Shower Stool

**Maximum  
User Weight** 

### General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.



### Intended Use

The Wide and Extra Wide Shower stools are intended to provide a comfortable, stable, seat for larger users who are unable to stand during showering. The Wide shower stool is designed to be suitable for use with most standard shower trays.

### Safety Precautions

- Do not exceed the maximum user weight limit of 254Kg (40st).
- Ensure an even floor surface to avoid rocking.
- Inspect product at regular intervals for wear and damage.
- Do not use if you see, or suspect a fault.

### Instructions for Use

**Height Adjustment:** To adjust the stool to the desired height;

1. Firstly remove the e-clips connecting the stools' legs and leg extensions.
2. Slide the leg extensions to the required position ensuring that all legs are equal in height.
3. Replace the e-clips ensuring that each clip is fully inserted into the tube.

### Cleaning Instructions

Clean regularly, wipe with a mild detergent/ disinfectant solution. Do not use abrasive cleaners or cloths on this product. Note: These stools are suitable to be cleaned in commercial high temperature washing machines.

These products are from a range of over 3000 daily living aids available from NRS Healthcare. To view the full range please visit our website or contact us, Tel: 0345 121 8111

Manufactured in the UK for  
**NRS Healthcare,**  
Sherwood House,  
Cartwright Way,  
Forest Business Park,  
Bardon Hill,  
Coalville,  
Leicestershire,  
LE67 1UB,  
United Kingdom.  
Tel. Head Office: 0330 100 0098  
Tel. Customer Services: 0345 121 8111  
Email:  
customerservice@nrshealthcare.co.uk

**www.nrshealthcare.co.uk**