

Product Technical Data Sheet 092

P23038 NUVO Adult Toilet Frame

Product Description & Intended Purpose


The NUVO Adult Toilet Frame is designed to assist users in lowering onto and rising from the toilet. It provides comfortable support for the user when getting onto and off the toilet.

Product Features


- Contemporary design with comfortable handles.
- Lightweight steel frame, powder coated white.
- Height and width adjustable.
- Designed to provide optimum stability.



TECHNICAL SPECIFICATION - ALL DIMENSIONS IN MM (INCHES)

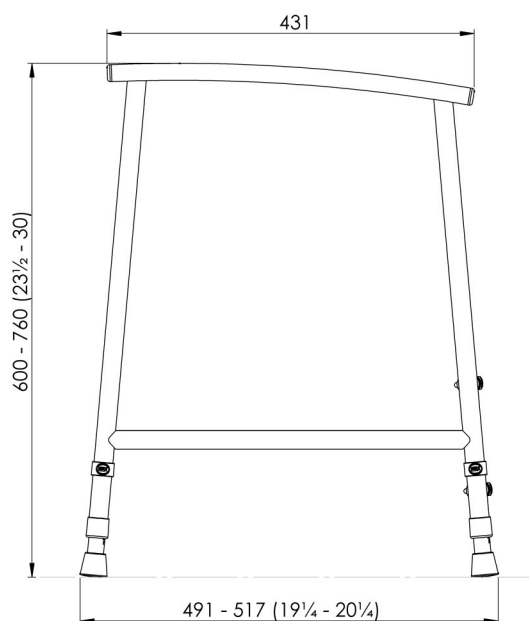

SAFE WORKING LOAD 190KG (30 STONE)

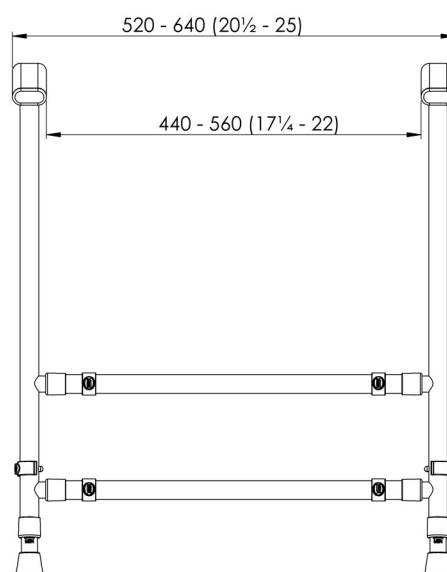

Installation Information: *(Note: for detailed information refer to User Instructions)*

Assembly

- Attach the cross bars to the side frames and insert 'e-clips' to secure.
- Insert the leg extensions by sliding into the frame legs to the correct height and insert an 'e-clip'.

To adjust the height:

- Remove the 'e-clips' and slide the legs to the desired position and re-insert the 'e-clips'.

To adjust width:

- Remove the 'e-clips' and slide the cross bars to the desired position and re-insert the 'e-clips'.

PTDS092

P23038 NUVO Adult Toilet Frame

Safety Precautions

- Ensure all four legs are set to the same height to prevent rocking and instability.
- Do not exceed the maximum user weight of the product, 30st (190kg).
- Do not attempt to climb on the product.
- Not suitable for use by children.

Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water.
- Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- Allow product to dry thoroughly prior to use/storage.

Care & Maintenance

- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Inspect the product regularly for signs of wear or damage, paying particular attention to the condition of the ferrules and e-clips.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Accessories / Related Products

P11668 - NUVO Plastic Fittings - 'e-clips' x 4, ferrules x 4, end caps x 4, anti-rattle collars x 4

FAQ's

- How do I know what height to set the frame at?

When the user is sat on the toilet frame, their elbows should rest on the arms of the toilet frame.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire. LE67 1UB UK