



Product	Nursing bench 2000, coated mesh cover, electrically height adjustable		
Item number	R8478000	Alle	
	VVS:777544480	750	
	NRF:7501961		
	EAN:5708590318523	\$130	
	HMI: 88223		
	NBS:N13:40,375	1600	
Colour	White	820 870	
Product description	Nursing bench 2000, powered, height adjustable 700 mm; from 300 mm -1000 mm, with wired hand control Foldable. Incl. safety rail and water collection tray with integrated outlet.  Outer measurements 900 mm x 1800 mm.  Excl. fixing kit.  See Technical datasheet (pdf) for information on fixings suitable for various wall types.		
Adjustability	Height adjustable		
Adjustment height	700 mm		
Maximum load	Maximum load: 175 kg. When the bench is loaded with 175 kg, each mounting bolt supports 23 kg This product has been tested statically with a load of 440 kg as the IEC 60601- standard requires testing with safety factor 2.5.		
Speed without load	Speed down mm: 20 mm/s		
Noise level	48 dB		
Materials	Powder-laquered stainless steel, mesh cover (PVC-coated for extra durability) and polystyrene.		
Ergonomy	Pressalit Care offers a range of high quality, fixed or power adjusted nursing benches designed to provide a stable platform suitable for changing adults and children in a variety of settings: schools, day care centres, institutions, hospitals and private homes.  The 2000 series nursing bench has a low start position, to make transfers to the bench easier. The washable, perforated mesh cover lets the water pass through to the water collection tray underneath.  The flexibility of the nursing bench provides for optimal working conditions for the carer as well as a high level of comfort and safety for the user.		





Product	A water collection tray with integrated water outlet prevents water from splashing on carers or the floor. The flexible hose attached to the centre mounted outlet of the water collection tray, can be connected to either an outlet in the floor or on the wall. The integrated gas cylinders enable the bench to be folded out of the way when not in use to maximise floor space in front of the unit.  Electric with wired hand control – the red buttons clearly indicate "Up" and "Down". The wired hand control comes with a holder for fitting to the wall. The stretcher of the bench is very easy to clean, and the stretcher cover can be unzipped for washing or easy replacement.	
Features		
Mains connection (V)	230	
Actuator (motor)	LA31 Careline, (Linak), 6000 N, IPX6	
Control box	CB9 Careline, (Linak), IPX6	
Wired remote control	HB3 (Linak), IP66	
Power usage	Power consumption by maximum load: 70 W Power consumption by standby: 2 W	
Warranty	Pressalit Care makes a commitment to remedy material and manufacturing defects for a period of three years from the invoicing date. NOTICE If the product is installed in an atmosphere heavy in salt or chloride, the warranty of the nursing bench will be reduced from the normal 3 years to 1 year. Salt and chloride have a corrosive effect on the products components and will cause a decrease in the life expectancy of the product. The damaging effects can, however, be reduced if the product is showered with tap water on a daily basis.	





Product	Nursing bench 2000, coated mesh cover, electrically height adjustable	
Cleaning	Nursing benches 1000 & 2000. The product is to be cleaned with hot water or a mild soapy cleaning agent. Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.	
	The following cleaning agents have been tested by Pressalit and may be used, as long as they are diluted according to the instructions on the container.	
	DISINFECTANTS: Clinimax Difficil-S Erisan OXY+	
	NON-DISINFECTANTS: Suma Star free D1 Taski 100 Taski 200	
	The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.	
CE	This product complies with the requirements of Council Directive 93/42/EEC concerning medical devices, as last amended by Council Directive 2007/47/EC of 5 September 2007. It also complies with all relevant requirements in the Council Directive 2006/42/EC (of 17 May 2006) on machinery and the Council Directive 2011/65/EC on RoHS.	





Product	Nursing bench 2000, coated mesh cover, electrically height adjustable		
Installation in different wall types			
	Concrete >B25  V8608 - Fixing kit, 6 coach screws Ø10 x 70 - plugs SX12. To be ordered separately.		
	Solid Brick >= Mz12  V8608 - Fixing kit, 6 coach screws Ø10 x 70 - plugs SX12.  To be ordered separately.		
	Solid Sand Lime Brick >= Hlz12  V8608 - Fixing kit, 6 coach screws Ø10 x 70 - plugs SX12.  To be ordered separately.		
	Aerated Concrete P4  V8658 - Fixing kit, 6 coach screws Ø10 x 70 - plugs GB14. To be ordered separately.		
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with wood  V8608 - Fixing kit, 6 coach screws Ø10 x 70 - plugs SX12.  NB! Plugs are not needed.  To be ordered separately.		





Product	Nursing bench 2000, coated mesh cover, electrically height adjustable	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with 22 mm plywood  Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant, re-enforced with steel plates (2 mm)  Wall fasteners (6 pcs) must be bought separately, for drywall it is possible to use e.g. fischer KD8 toggles or similar.	
	Drywall (2 x 12,5 mm) water-resistant  Mounting in this wall type is not recommended - risk of fatal injury to persons and walls.	

It is the responsibility of the installer to ensure that suitable wall fasteners are used.







www.pressalit.com