

USER INSTRUCTIONS

BATHROOM STEP

M99515

Bathroom Step

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

To assist the user while transferring into or out of the bath, by reducing the step-over height.



Single Step Use: Unpack the step, ensure it is clean and dry, remove the protective film from the back of the self-adhesive mat and place accurately on top of the step. Ensure the mat is in the recess on the top of the step and there are no creases, press down firmly to secure.

Multiple Step Use: Stacking Options (Steps purchased separately).

- Single: 4" (100 mm)
- Two Steps: 6" (150 mm)
- Maximum 3 High: 8" 200 mm).

2. SAFETY

Warning: The maximum user weight must not be more than 190 kg (30 st).

The maximum step height is 8" (200 mm) using 3 steps (purchased separately).

Do not stack more than 3 steps together.

Always use extreme caution when getting in or out of a bath.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

3.1 Contents:

1. 100 mm (4") Step (complete with four assembled caps) x 1
2. Anti-Slip Mat (Self Adhesive backed) x 1
3. User Instructions x 1

4. ASSEMBLY

No tools are required to assemble this product. (However, a flat headed screwdriver, or similar, would help to remove the caps if the steps are to be assembled in a multiple stack).

There are a number of options and combinations the steps can be used for.

A single step provides a step height of 4"(100 mm).

Note: The Anti -Slip mat is placed on the top step only. Mats supplied with the other steps should be saved, to be used later as spares.

Stacking assembly: The top step should be assembled as described in the single step use section.

For all other steps used in the stack, remove the four caps as shown in Figures 1 & 2.

Note: a flat bladed screwdriver may help to do this. Please use caution if tools are used.

Remove all four caps.



Figure 1:



Figure 2:

Step combination: If the steps are used to form a stair step, shown below, then the steps should be linked as shown in Figures 3, 4 & 5.

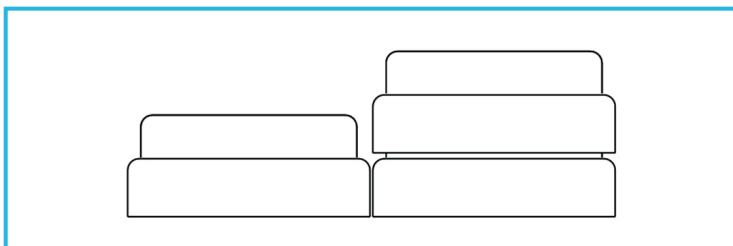


Figure 3: Stair Step Combination



Figure 4:

Using the hooks on one side of the step, link to the opposite side on the other step.



Figure 5:

Ensure the steps are correctly linked as shown above before use.

5. **CLEANING & MAINTENANCE INSTRUCTIONS.**

Clean regularly, wiping down with a soft cloth and a mild detergent/disinfectant solution.

Do not use abrasive cleaners.

This product can be autoclaved up to 80°C.

Inspect on a regular basis the stability and rigidity of the product.

6. **PRODUCT DISPOSAL**

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

7. **PRODUCT GUARANTEE**

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk