

Do Not Use In Water

Tumble Forms are not water toys. They are designed to take in and exhaust air for the cushion-like effect. Naturally water will also be soaked up (as in a sponge) through the vent hole if the cushion is submerged in water.

Outdoor Use

Tumble Forms are designed to be used indoors. However, if they are used out-of-doors, care should be taken not to let them get wet. If the cushions do accidentally absorb water through the vent holes they will dry out best by squeezing and pressing them. Any remaining dampness will disappear after a few days by exchange of dry air through the vent holes.

Cleanability

Tumble Forms covering may be cleaned with household or industrial grade detergents or all-purpose liquid cleaners. Permanent markers and ballpoint pen markings cannot be removed. However non-permanent ink, wax, crayon, oil, grease, dirt can be scrubbed and wiped clean with a wet rag. (Do not soak the cushion – it will get waterlogged).

Use and Care Instructions for Tumble Forms

Tumble Forms Neuro-Developmental Treatment modules and adaptive positioning shapes were especially developed to meet the needs of therapists and teachers working with neurologically challenged individuals. They combine therapeutic design, durability and color to help solve problems of positioning and maintaining the individual with special needs.

Made of soft, flexible yet tough materials, the modular units have exceptional resistance to tearing and abrasion. With the normal care and respect that you would give any cushion, Tumble Forms will give many years of active service.

New Cushion Odor

Residue odors from the manufacturing process linger within the foam cell structure. This will disappear after a few days' use, as fresh air replaces the air inside as the cushion is pressed.

Tumble Forms²

**With Anti-Microbial
Protection**

Made in the USA

Use and Care Instructions

Congratulations!

You have just bought one of today's most revolutionary products, designed for those with special needs. Tumble Forms combines innovative design with technology intensive fabrication to provide a high degree of durability and attractiveness. Moreover, the unique seamless Tumble Forms upholstery provides the ultimate in moisture protections.

So you may enjoy the maximum usage from your Tumble Forms, please read the use and care instructions inside.

Upholstery Variations

Because of the unique handcrafted Tumble Forms Process, minute differences may occur in appearance of protective upholstery. What appear to be tiny craters are not a defect, but are due to variations in atmospheric conditions during the manufacturing process. This in no way affects the strength and moisture resistance of the upholstery.

Vent Holes

Each Tumble Forms shape has at least one vent hole. This has been heat stamped through the covering, and is necessary to equalize air pressure in the foam with atmospheric pressure. Since temperature changes and consequent atmospheric pressure changes may be encountered during shipment, these vents maintain the proper shape and resiliency of your Tumble Forms Module.