Technische Daten –**Technical Data****Art- Nr . 13082****Art.No. 13082**

Durchmesser Sauger – *Diameter suction cups:*

85 mm

Gesamtlänge – *Length over all:*

500 mm

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100 mm

Abstand zur Wand – *Distance to wall:*

50 mm

Farbe – *Colour:*

Weiß/orange – *white/orange*

Gewicht – *Weight*

0,5 kg

Material Griff – *Material grip:*

Kunststoff PP -High-quality plastic PP

Material Saugteller – *Material suction cups:*

NBR

Material Innere Mechanik – *Material inner parts:*

VA (stainless steel)

Garantie –**Warranty**

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www.drivedevilbiss.co.uk
www.drivemedical.de

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Indication

Ideal for people who require assistance with showering and/or bathing on a daily basis. Can be used on suitable flat and smooth surfaces (e.g. tiled surfaces).

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Product contents

Suction grab bar with 2 suction cups, ready to use.

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Clean the vacuum suction discs and the surface thoroughly. Press the rubber discs firmly to the surface. The tilting vacuum levers must be loose at this point and the whole of each rubber disc must lie on the surface.

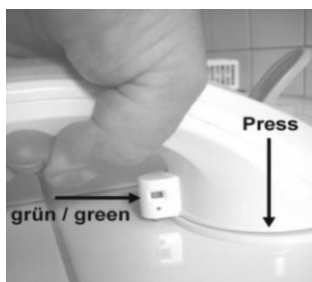
Ensure that the rubber discs are completely on the surface, and that the discs do not cover any grouting lines or other joints (see pictures below).

Firstly, press the grab bar firmly on the surface to which it is to be secured ensuring that the visual aid for suction is green (photo 1).

To secure the suction cups attached to the end of the bar, press the lever from an up-right to a flat position (photo 2). Do this for both sides of the grab bar.

Before each use, test the bar is secure. If the bar is not secure, release the bar and reattach again.


If the required suction power has not been achieved, the bar should not be used !

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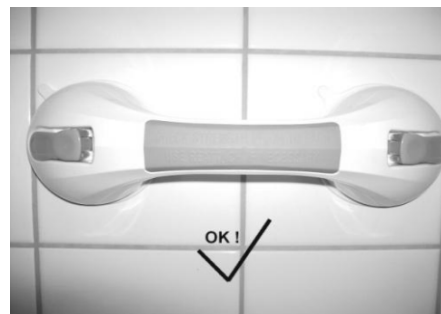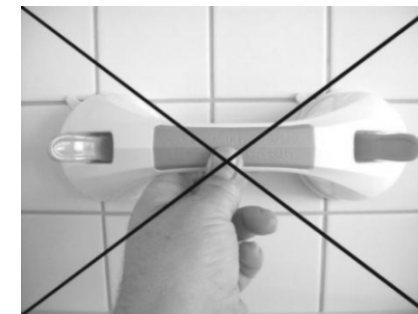
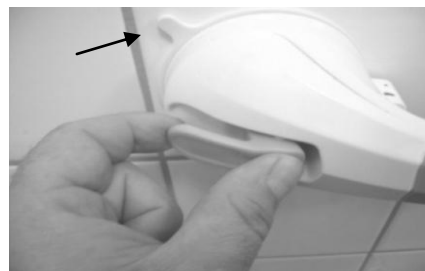
Removal instructions: Release bath latches. Pull the flap to remove the suction cups.

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Safety Instruction

Attention

*This product is designed to provide support. It is not designed to hold a person's entire body weight. Before using the grab bar, clean and dry the area where the grab bar is to be attached, **and read the instructions carefully.***

Be sure to test the stability of the grab bar before every use.

Always make sure that the grab bar is securely attached before each use and always reattach if loose.

Do not use the suction grab bar if it is damaged

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Cleaning

Always keep the rubber discs free from oil and dust. Clean the rubber discs with standard household detergents (no acetic cleaner).