

User Instructions

Dovedale Folding Commode



Dovedale Folding Commode S35000











Read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

Intended Use



For use as a portable, folding commode by people who have difficulty accessing or using standard toilets. Use the commode in dry areas only - not wet-rooms or showers.

Symbols


Symbols used on the product and in these instructions.

	Manufacturer		Batch Reference Code		Consult the instructions for use
	Item/Catalogue number		Important Note		Recycling Symbol
	Maximum User Weight		Warning		

Safety



Do not exceed the maximum user weight of 130 kg (20 st).



Inspect the product before use. Contact the service provider if you see any damage.
Wipe clean after every use.

Before use:

- Put the commode on a sturdy, level surface.
- Unfold the commode correctly and lock both catches.
- Check that the commode is stable.

Do not:

- Use in wet areas such as showers or wet-rooms.
- Stand on the chair or sit on the armrests.
- Lean back on the seat.

Unpacking

Remove the packaging and check all parts carefully.



Contact your supplier immediately if parts are missing or damaged. Do not try to assemble.

Contents



Chair unpacked in it's folded state.



1. Chair Assembly
2. Potty and Lid

Assembly and Use

1



Unfold the commode.



The Potty does not fit when the chair is folded.

2



Secure the catches.



3



Remove the potty lid and slide the Potty in from the back of the chair.



The Potty does not slide in from the front of the chair.

4



Chair assembled, ready for use.


After Use

1




Remove the potty from the back of the chair and refit the lid. Use the wire handle to carry the potty away for emptying and cleaning. Clean the seat if necessary.

2

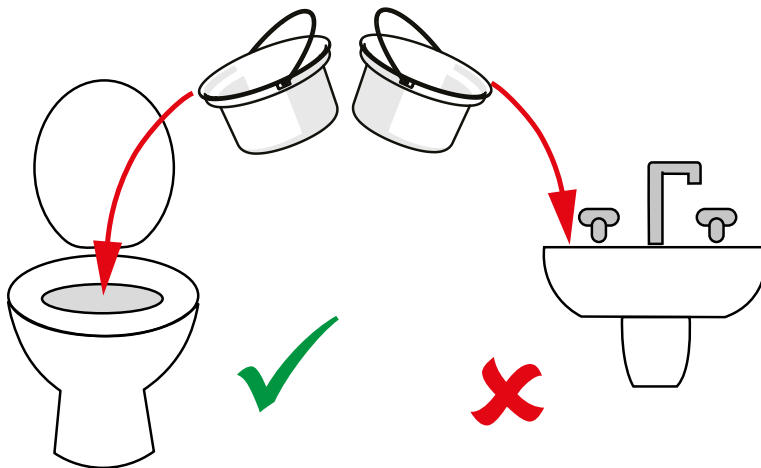

Dispose of potty contents correctly.

3

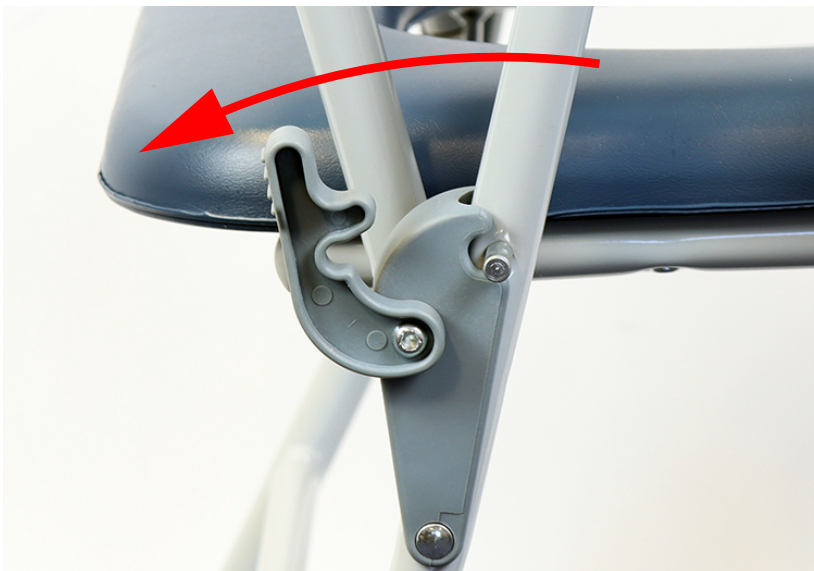

Release the catches and fold the chair for storage.

Maintenance

Inspect the product at regular intervals and before re-use for signs of general wear or damage.
Report to the supplier any suspected faults or damage and take the product out of use immediately.
Check the fixings at regular intervals.

Cleaning

Clean regularly using a general purpose, neutral detergent and warm water. Check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

Allow product to fully dry before using and storage.

Accessories & Spare Parts

- M99096 Potty Kit Spare

For more information, see the end of this document for our contact details.

Product Disposal

Thoroughly clean this product before disposal and recycle where possible.

This product is made of wood, metal, polyurethane and plastic. It can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.



Recycle any cardboard packaging.

Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us. See the end of this document for our contact details.

If you have experienced an injury, contact us but feel free to contact the Medical and Healthcare products Regulatory Agency at: devices.regulatory@mhra.gov.uk

We are a leading supplier of community equipment and daily living aids.
Please visit one of our websites to see our full range of products.

NRS Healthcare. Sherwood House,
Cartwright Way, Forest Business Park,
Bardon Hill, Coalville,
Leicestershire, LE67 1UB, UK

Tel: +44 (0)345 121 8111
Email: customerservice@nrshealthcare.co.uk
Web: www.healthcarepro.co.uk
or www.completECAshop.co.uk