

Spa Range

UML1000 Adult Male Urinal Bottle 1000ml

Long Life Re-usable Plastic Male Urinal Bottle for use in Nursing, Rehabilitation, Elderly Care, Community Care and Personal Home Use



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



Product Benefits

- Lightweight, strong and durable.
- Secure fitting cap to prevent leaks and spills and reduces unpleasant odours and risk of cross infection.
- Graduated with clear, easy to read measurements.
- Easy grip handle - particularly suitable for those with limited hand movement.
- Hollow handle for improved cleaning results.
- Can be cleaned in automatic washer disinfectors or hand washed.
- Smooth material surfaces ensure ease of use and ease of cleaning.

Product Specification

- Male Urinal Bottle - Graduated, 1000ml
- Pack Size 50
- Registered as a Class 1 Medical Device (CE Marked)
- Complies with following Standards:
 - **ISO 9001:2008**
Quality Management System
 - **ISO 13485:2003**
Medical Devices - Quality Management System
 - **BS3215:1995**
Re-usable Portable Urinals
- Manufactured from high density Polyethylene

Guide to Using Spa Products from Warwick SASCo

Usage

- For male use only.
- Can be used in supine, seated or standing position.
- Close cap immediately after use to prevent spillage and odours.
- Empty after each use.

Cleaning

- Product should be washed before first use. Wash in hot water with detergents and disinfectants and allow to dry naturally.
- Can be cleaned in automatic washer disinfectors compliant with the requirements of ISO15883. Also suitable for hand washing using detergents and disinfectants.
- Spa Products are designed for repeated use - guaranteed for 1000 washes.
- Scouring and abrasive cleaners will reduce the performance of products.

Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.