

## PRODUCT INSTRUCTION SHEET



### REFERENCE

172 Uppingham combined shower & commode chair  
172-LONGARMS- Uppingham combined shower & commode chair  
 (LONG ARMED VERSION)  
172WH - Footrest

### DESCRIPTION

Manufactured from peel resistant, plastic coated steel tubing, this very robust stylish product is supplied with a comfortable PU Cellular Moulded gap-fronted seat, backrest, over seat and padded hinged armrests. The unit features a pushing handle and 4 braked nylon castors (125mm diameter) and commode bowl. Also included is a rota molded plastic detachable footrest with 2 height positions. The footrest simply clips to the frame and is retracted when not in use. All accessory components are stainless steel for product longevity with re-usability and re-issue in mind.

### TECHNICAL SPECIFICATIONS

Width between arms:	21" (54cm)
Seat Height:	20" (50cm)
Seat Width:	17" (42cm)
Seat Depth:	17" (42cm)
Aperture Size	11.5" (33cm) x 8.5" (22cm)
Depth of Pot	7"
Height from floor to top of commode seat:	20" (51cm)
Height from floor to bottom of commode seat:	18" (45.7cm)
Overall width:	25" (64cm)
Overall depth:	21" (52cm)
Height to top of pushing handle:	40" (102cm)
Weight limit:	160kg (25 Stone)
Length of Arms (from back of frame):	13" (33cm)
	(Long arm version 16" (41cm))



### USAGE REQUIREMENTS

1. A thorough risk assessment should be carried out when prescribing this product.
2. Check all castors are securely fitted before, during and after use.
3. Ensure the spindle bolt which holds the lift up arms is tightened at all times.
4. The chair is designed to be used on level surfaces only.
5. If using a moulded footrest make sure the sides are properly located on the bottom struts of the frame, and move freely from front to rear and vice versa.
6. The footrest has 2 height positions; the higher position must be used for children or smaller adults only to prevent tipping off the unit when in use.
7. When in use we advise that all 4 brake castors are secured at all times and the first 2 castors have the brake pedal facing the front, and parallel with the side of the chair. This will prevent the chair from tipping if the client leans forward.
8. The chair is designed to withstand temperatures provided by the latest disinfecting commercial washing machines. Otherwise, use a soft cloth dipped in neutral detergent and warm water. Do not use harsh abrasive materials. To disinfect, clean as above and wipe over with a solution of one thousand parts per million chloride disinfectant-rinse and dry.
9. This product is intended for use in a wet room / shower. It is not intended for use outside and should not be immersed in water for long periods of time. Ensure the product is dried thoroughly after use.