

# USER INSTRUCTIONS SEAHORSE PLUS CHEST & BUTTERFLY HARNESS

N72139	Seahorse Plus Chest Harness (Small)	N72140	Seahorse Plus Chest Harness (Medium)
N72723	Seahorse Plus Chest Harness (Large)	N72152	Seahorse Plus Butterfly Harness (Small)
N72164	Seahorse Plus Butterfly Harness (Medium)	N72759	Seahorse Plus Butterfly Harness (Large)

## ⚠ General Safety Warning ⚠

**Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.**

### INTENDED USE

Optional Poziform™ Chest & Butterfly Harnesses have been designed for users requiring more secure positioning when using the Seahorse Plus Toileting and Shower Chair.

The Butterfly Harness is particularly suitable for female users.



**Chest Harness**

**Butterfly Harness**

### SAFETY

**Warning:** Only use the correct sized harness for the chair.

**Warning:** Check regularly for signs of wear or damage and replace where necessary.

**Warning:** The chest and butterfly harnesses should be used alongside the NRS Lap Strap (N99089, N99091 or N99108).



### UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

### FITTING THE CHEST & BUTTERFLY HARNESSSES


The chest and butterfly harnesses are supplied separately as optional accessories. Both harnesses are fitted in the same method.

Make sure you have the correct size for the chair.

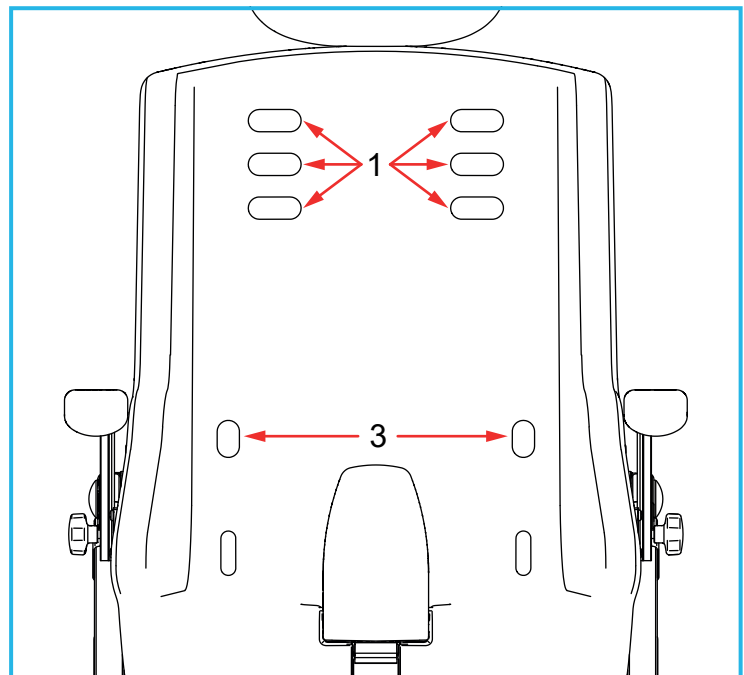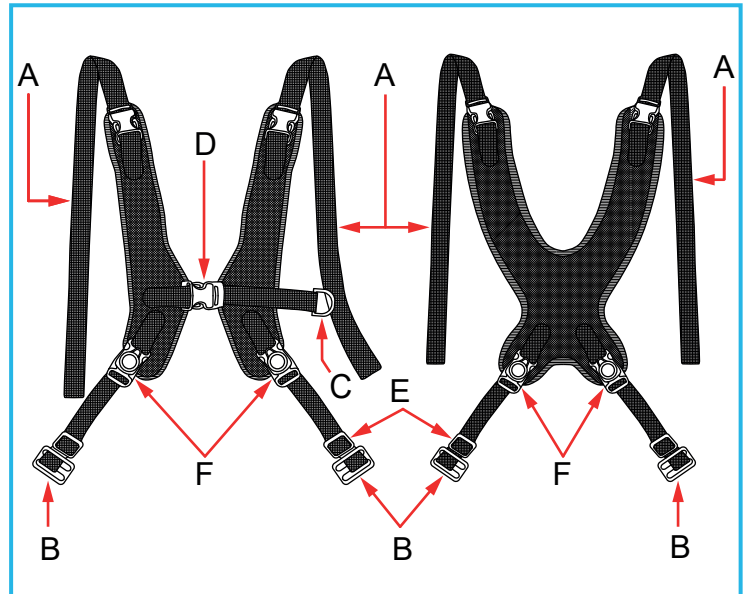

**Chair Front**

1. Feed the upper straps (A) through the upper slots in the seat / liner (1). There are three pairs of slots, allowing a different height setting for the harness.

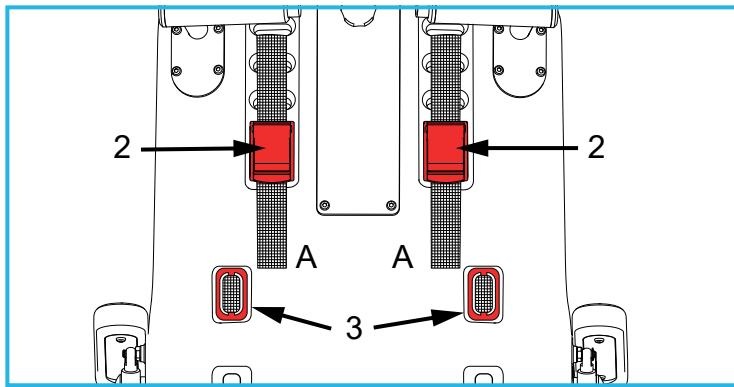

2. Feed the upper straps (A) down through the cam-locks (2), then lock the cam-locks closed.

**Note:** Leave plenty of slack at this stage.

3. Feed the buckles (B) on the lower straps through the lower slots in the seat / liner (3), by rotating the buckles 90 degrees and pushing firmly.
4. Align the buckles (B) to fit flush at the rear of the seat (as shown), then pull the straps tight from the front to make sure they are fully secure.
5. Use the adjustments sliders (E) to adjust the bottom straps.

## USING AND ADJUSTING THE HARNESES

**Warning:** You must fit and secure the user with the lap strap before securing the harness.

### Using the Chest Harness

1. Unclip the central buckle (D) and loosen the front strap (C), then sit the user in the chair.
2. Reclip the central buckle (D), then pull the front strap (C) to secure the user.
3. Tighten the upper straps (A) from the rear of the chair, by opening the cam locks (2) and pulling down on the straps (A).
4. Close the cam-locks (2) to secure the upper straps (A).

### Using the Butterfly Harness

1. Unclip the lower strap buckles (F) from the harness.
2. Lift the harness and sit the user in the chair.
3. Lower the harness over the user and reattach the lower strap buckles (F).
4. Tighten the upper straps (A) from the rear of the chair, by opening the cam locks (2) and pulling down on the straps (A).
5. Close the cam-locks (2) to secure the upper straps (A).

**Warning:** Make sure the harnesses are correctly adjusted for the user every time the user is seated. Incorrectly adjusted harnesses can result in the user sliding forwards in the chair.

## MAINTENANCE

Check all straps and harnesses for any signs of damage to the webbing or to the buckles. Damaged items must be replaced immediately.

Please contact NRS Healthcare Customer Services to report a fault or book an inspection by a service technician.

## CLEANING INSTRUCTIONS

- Clean with sponge and warm soapy water.
- Do not use bleach or solvents.
- Do not steam clean, tumble dry or dry clean.
- Allow product to fully dry prior to use/storage.

## PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from plastic and fabric and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address given.

If you have experienced an injury, contact us but feel free to contact the Medicines and Healthcare products Regulatory Agency at:

[devices.regulatory@mhra.gov.uk](mailto:devices.regulatory@mhra.gov.uk)

**NRS HEALTHCARE** Sherwood House, Cartwright Way, Forest Business Park  
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)