

USER INSTRUCTIONS

ADJUSTABLE TOILET SAFETY FRAME

Q36374

Adjustable Toilet Safety Frame

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

Maximum User Weight



INTENDED USE

The Adjustable Toilet Safety Frame is designed to provide additional support and safety for people with low mobility when they use the toilet. Made from strong 1" diameter aluminium tubing, the Adjustable Toilet Safety Frame is both durable and reliable and significantly increases safety in the bathroom by helping to negate the risk of slips and falls. This frame fits most toilets using the existing toilet seat fixings and is corrosion resistant to ensure its longevity. In addition, the Adjustable Toilet Safety Frame is height and width adjustable.



Figure 1: Toilet Safety Frame

SAFETY

WARNING: The maximum user weight must not be more than 113Kg (17st).

WARNING: Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance.

CAUTION: Please inspect all parts carefully for cracks or breaks before use.

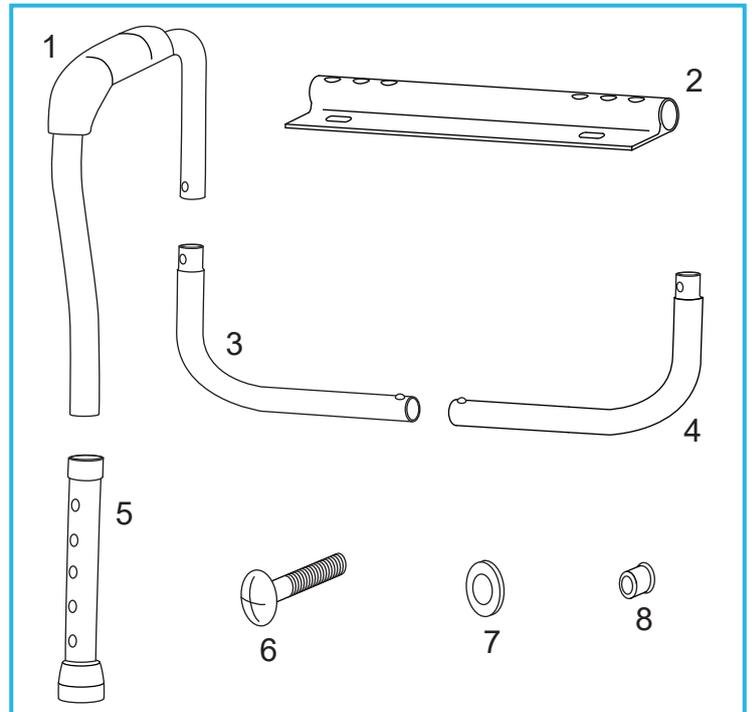


Figure 2: Parts

UNPACKING


Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try to assemble.

Contents:

1. Handrail Tube x 2
2. Toilet Frame Bucket Tube
3. L-shape Tube RHS
4. L-shape Tube LHS
5. Leg Tube x 2
6. Screw x 2
7. Washer x 2
8. Nut x 2

ASSEMBLY/INSTALLATION

1. Attach the Toilet Frame Bracket Tube:

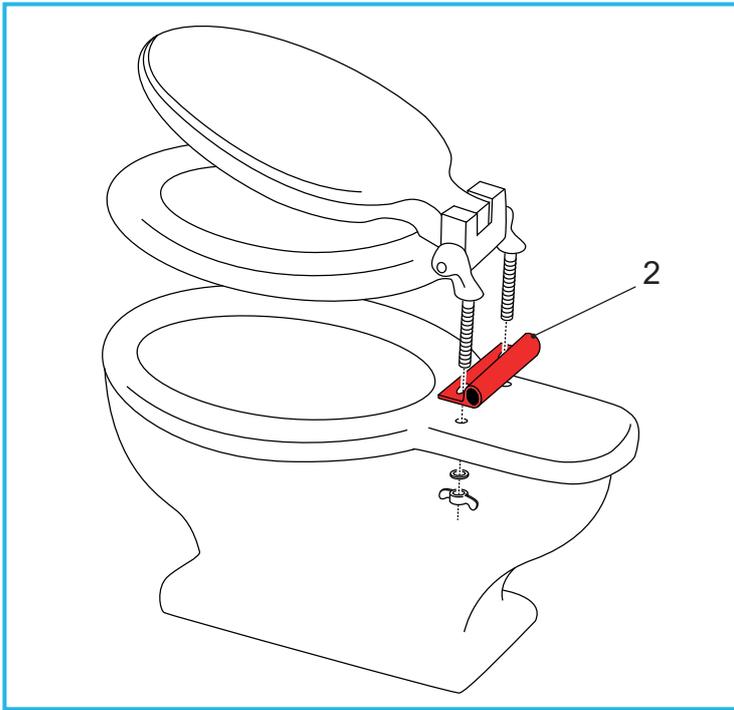

Remove the toilet seat and lid.

Make sure the toilet, seat, lid and fixings are clean and free of obstructions before fitting.


Place the toilet frame bracket tube (2) on the rim of the toilet and line up the holes.

Note: Make sure the bracket tubing is located at the back as shown.

Position the toilet seat and lid over the bracket and line up the holes

Fix the toilet seat, lid and frame bracket tube using the existing toilet seat fixings.

2. Fit the L-shape Tubes to the Handrail Tubes:

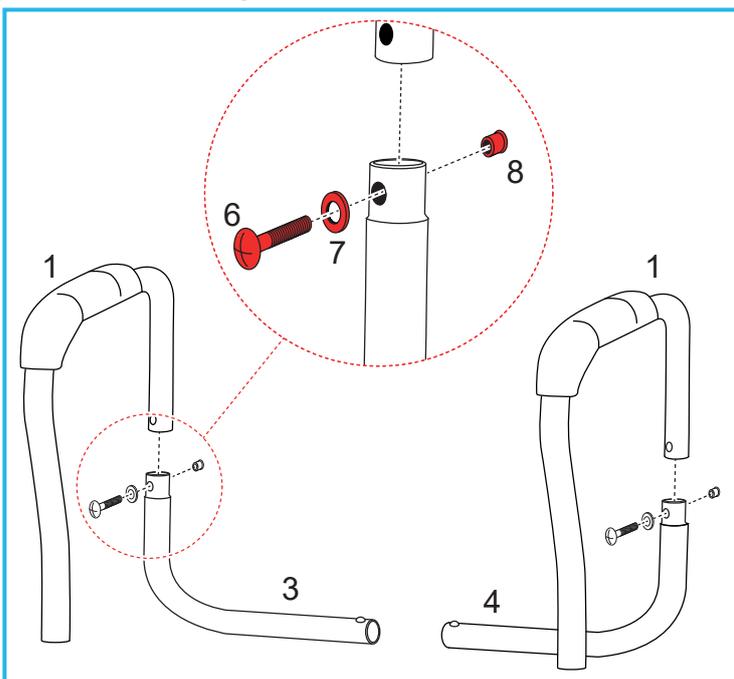


Note: The L-shape tubes are specific to the left and right hand side of the product.

Insert the L-shape connecting tubes (3 & 4) into the handrail tubes (1).

Secure the tubes using the screws (6), washers (7) and nuts (8).

Note: Make sure the fixings are fully tightened before use.

3. Insert the Leg Tubes into the Handrail Tubes:

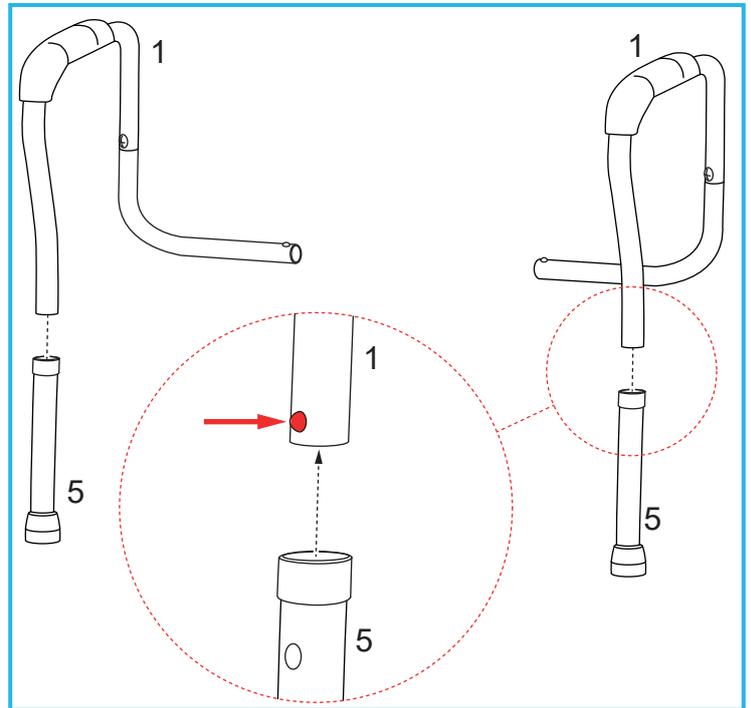


Press and hold the button clip on the handrail tube, then slide the leg tube (5) onto the handrail tube (1) until the button clip clicks into place.

Note: Make sure that the button clip has fully engaged the hole.

4. Adjust the Leg Height

The leg tubes must be adjusted to suit the height of your toilet.

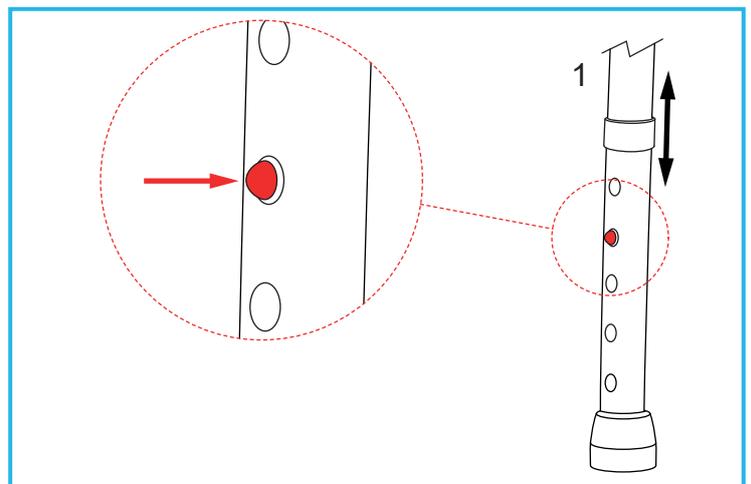


Press and hold the handrail button clip and slide the leg to the desired position until the button clip clicks into place.

Note: Make sure that the button clips have fully engaged the holes.

Note: Make sure that both legs are set at equal height and in contact with the floor.

Note: The armrests should be level once fitted to the toilet frame bracket tube.


5. Fit the Handrail Assembly to the Bracket Tube:

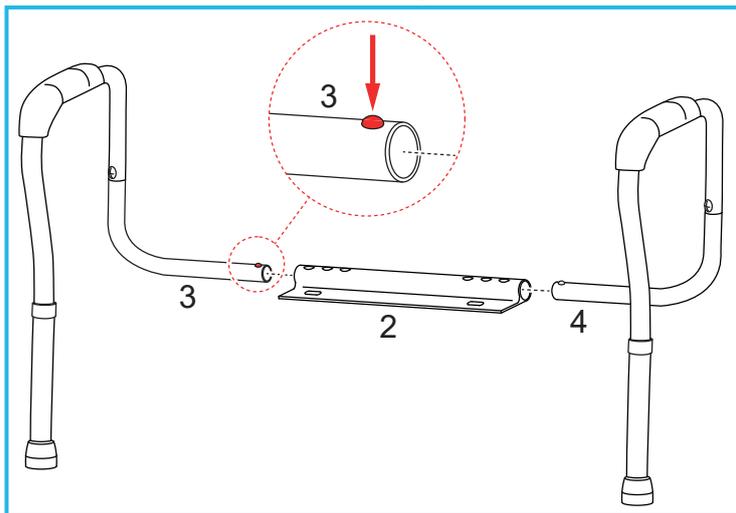

Press and hold the button clips on the L-shape tubes (3 & 4), then slot them into toilet bracket tube (2) until the button clips click into place.

Note: Make sure that the button clips have fully engaged the holes.

6. Adjusting the Handrail Width

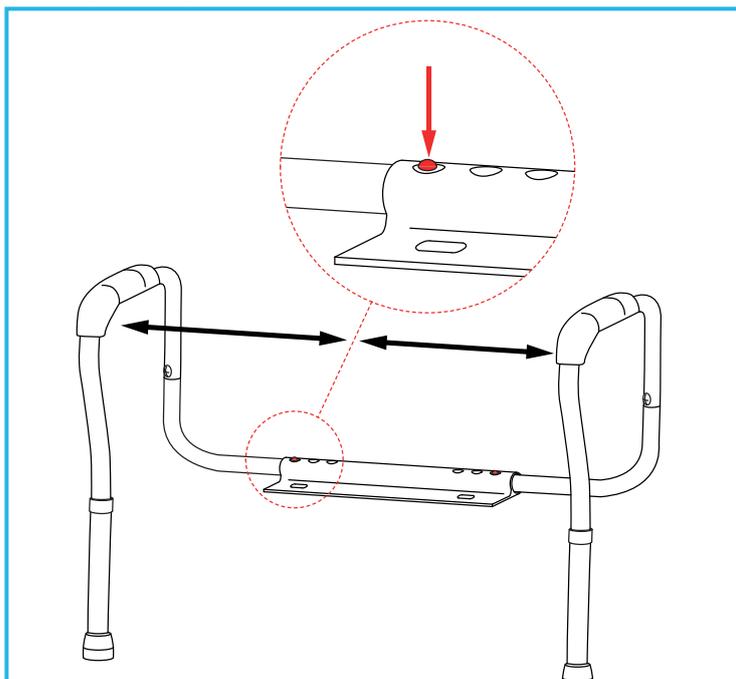

Press and hold the handrail assembly button clip, then slide the handrail to the desired position until the button clip clicks into place.

Repeat for the other side.

Note: Make sure that the button clips have fully engaged the holes.

Note: Make sure that both handrails are set at an equal distance.

Warning: Do not try to adjust the product during use.

CLEANING

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 80°C.

Allow product to fully dry prior to use/storage.

MAINTENANCE

Inspect the product at regular intervals and before re-use for signs of general wear or damage.

Any suspected faults or damage should be reported to the supplier immediately and the product taken out of use.

Check the ferrules are not worn, damaged or missing before using the product. Replace if necessary.

PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardonia Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

Q36374-UI-1