

Product Technical Data Sheet 040

2 in 1 Bed Rail

Product Range

M48192	2 in 1 Bed Rail
N71986	2 in 1 Contour Bed Rail

Product Description & Intended Purpose

The 2 in 1 Bed-rail provides a stable support to assist clients getting into and out of bed. When in bed the Bed Rail can be used to assist raising to a sitting position or repositioning when lying.

Designed for use with divan beds (except sprung divan base), most slatted domestic beds and is suitable for all bed sizes from single to king-size. It can also be fitted to super king-size beds with additional straps. Not suitable for profiling beds.



Product Features

- Ergonomic handle is comfortable to grip.
- Flat base which is unobtrusive to lie on.
- Reversible handle and base allows use on a variety of bed styles.

Technical Specification

Dimensions (mm):

- Length: 615 (24")
- Width: 335 (13")
- Base Thickness: 15 (½")
- Height from bed (Divan): 500 (20")
- Height from bed (Slatted): 550 (21½")
- Product Weight: 3.5kg (7¾lb)

Maximum User Weight: 160kg (25st)

Safety Precautions

- To avoid possible head entrapment, position the Bed Rail at least 320mm (12.5") away from both headboard and footboard.
- Always ensure that the mattress is as close as possible to the Bed Rail. Any gap could allow the client to become wedged in-between.
- Do not exceed the maximum user weight of the product 25st, 160kg.
- Always use the securing straps provided when fitting to a divan bed.
- Regularly check the straps are tight and the device is stable and firmly fitted.
- Not suitable for use by children.
- Do not fit this product to profiling beds or sprung divan beds.

PTDS040

2 in 1 Bed Rail

Installation Information

- Product is delivered flat packed and requires assembly before use.
- Product assembly is dependent upon bed style, please refer to user instructions for details.
- Assembly time is approximately 5 minutes.
- No specialist tools or skills are required.

Cleaning

- Wipe product clean with a soft cloth moistened with soapy water. Ensure product is rinsed and dried fully before storage or refitting.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water. Do not use bleach to wash the straps.

Maintenance

- Check regularly for signs of wear or damage.
- Remove from use immediately if there are signs of wear, damage or if the Hand Rail is excessively loose in the Bed Rail Base.
- Check that all fixing straps are tight and the Bed Rail is correctly positioned.
- Check there is no gap between the Hand Rail and the mattress.
- Check both 'e-clips' are fitted correctly and are secure.

Accessories / Related Products:

- L75930 - 2in1 Bedrail Strap
- M84780 - Slip Resistant Netting (white)

FAQ's

- How long is the length of the strap which goes around the bed?
The length of strap is 4.5 metres.
- Where should the rail be positioned on the bed?
There should be a distance of at least 320mm from the headboard to the handrail to prevent entrapment.
- Can this be used on profiling / adjustable beds?
No this is not suitable for use on profiling / adjustable beds.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.