

Product Technical Data Sheet 060 Pedal Exerciser with Digital Display

Product Range

M37352	Black & Silver (pictured)
P01432	Black & Green

Product Description & Intended Purpose

Designed for low impact rehabilitation and exercise this Pedal Exerciser is NOT a suitable alternative to a full size gym exercise bike or similar. It is ideal for those with reduced mobility or those who wish to complete a little gentle exercise. It incorporates a digital display to enable users to view cycles completed, RPM, duration and calories burned.



Product Features

- Digital Display of exercise data.
- Toe straps on pedals.
- Foldable frame design for storage and transport.
- Adjustable resistance.

Technical Specification

Dimensions (mm)

Folded: 340 x 370 x 190 (13¼" x 14½" x 7½")

Unfolded: 480 x 370 x 270 (18¾" x 14½" x 10½")

Pedal: L 100 x W 45 (4" x 1¾")

Strap Height: max 70 (2¾")

Crank length: 90 (3½")

Product Weight: 2kg (4.5lb)

Safety Precautions

- CAUTION! As friction is increased, or during prolonged periods of vigorous exercise, the pedals and housing may become warm to the touch. Stop using the exerciser and allow to cool.
- Do not attempt to stand whilst using the exerciser.
- Ensure the exerciser is locked in the open position prior to use.
- Always place the exerciser on a flat, 'non-slip' surface. Never use on a wet floor or in a bathroom environment.
- Not suitable for use by children.

PTDS060

Pedal Exerciser with Digital Display

Installation Information: *(Note: for detailed information refer to User Instructions)*

- The product is supplied pre assembled. It simply requires unfolding.
- No tools or specialist skills are required to install, adjust or use the product.
- Installation time is approximately 5 minutes.

Cleaning

- Wipe clean with a damp cloth. Take care when wiping the digital display area.
- Please check the suitability of the cleaning product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Allow the product to dry thoroughly prior to use.

Care & Maintenance

- Inspect the product regularly for signs of wear, pay particular attention to the condition of the pedals and rotating mechanism.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Replacing the Battery:

- To access the battery, insert a small, flat ended screwdriver into the slots on both sides of the digital display to release the clips.
- Lift the display out of the housing. *Take care not to damage the connecting wires.*
- Replace the battery and re-insert the display. *Ensure the clips 'click' back into position securely.*

FAQ's

- What sort of batteries does the exerciser require?
A single watch style battery is needed.
- What is the resistance like? Is it as easy as pushing air or hard work like pushing a mountain?
The level of resistance can be increased or decreased depending upon the ability of the user.
- Is it possible to adjust the height of the exerciser?
No the exerciser has a fixed height but you can adjust your position by moving your position on the chair or the chair itself.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK