

M48787 Shower Seat with Legs

M53552 Shower Seat with Legs & Padded Seat

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

This wall mounted folding shower seat range, with legs, is intended to be used in a shower to provide a stable, seated support, where a user is unable to stand for long periods (i.e. during showering). It can be folded up against the wall when not in use.



M48787



M53552

2. SAFETY

This Shower Seat should be correctly installed by a suitably qualified, competent person.

Only install the product in an area where the mounting surface is structurally sound. (Strengthening modifications to the fixing surface may be required).

This product must be secured to the wall using fasteners which are appropriate to the structural base material.

DO NOT install in fibreglass / acrylic shower enclosures with insufficient stiffness and strength.

The maximum user weight for this product is 160kg (25st) based on the product being fitted to a suitable surface using appropriate fixings and fitted by a competent person.

Waterproofing is recommended, and all fixings used should be non-rusting. Sealant should be applied around wall contact points to avoid water ingress.

Inspect the product at regular intervals for wear and damage, do not use if unstable or the rubber feet are damaged.

Maximum User Weight



3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

4.1 Fitting and Height Adjustment

1. Identify the correct height and position for the seat within the shower.

Warning: When choosing the position, check there are no concealed electrical cables or pipe work near to where the fixing holes are to be drilled.

2. Using the e-clips and external leg holes, adjust the leg extension tubes to the correct height for the client. For finer adjustment, the legs may be adjusted individually by unscrewing the threaded feet and locking into position by tightening the nut.
3. With the seat in the down position, 90 degrees to the wall, and the legs vertical, with the feet flat and secure to the shower base, mark the fixing hole positions for the wall mounting bracket on the wall.
4. Using appropriate, non-rusting screws and fixings, secure the mounting bracket to the wall.
5. Check the seat can be raised and lowered without interference, and that the feet are securely in contact with the shower base.
6. Check seat is firm and secure.

5. IN USE

- Before using, test with both hands, to check the seat is securely fastened to the wall.
- To lower; pull down on the seat edge.
- Ensure both legs are level and in contact with the shower floor before sitting.

- Sit centrally on the shower seat.
- After use, you may lift the seat back up against the wall.
- Do not stand on the seat.
- Do not lower, or raise, the seat using the legs, as this may cause damage.

6. MAINTENANCE

- Inspect the seat at regular intervals, ensure the product is stable and rigid.
- Check all fixings are fully tightened and in good condition.
- Check ferrules monthly for signs of wear.

7. CLEANING INSTRUCTIONS

The frame & seat may be cleaned using a non-abrasive cream cleaner or detergent solution.

8. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: 0345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk