

User Instructions

Medeci Systemware

General Safety Warning

Please ensure these instructions are fully read and understood before use

Intended Use

Medeci Systemware offers flexible solutions to a wide range of eating and drinking situations. Made from lightweight, high impact polymers, the bright contrasting colours make the range especially suitable for the visually impaired. The deep inner wall of the plate aids people who need to scoop up their food with one hand. The Medeci cup features two detachable, easy to hold, handles and a range of tops to reduce the risk of spillage and help people who have difficulty with conventional cups.

Unpacking/Use

Remove all items from the packaging, check for any visible signs of damage or fault. If you suspect the product is not fit for purpose, do not use and contact your supplier immediately

Dimensions:

Cup:

Height: 98mm (3³/₄"")
 Diameter: 85mm (2¹/₂"")
 Capacity: 280ml.

Plate:

Outer Diameter: 240mm (9¹/₂"")
 Internal Diameter: 180mm (7"")
 Internal Depth: 25mm (1"")

Care and Maintenance

- Wash using normal domestic procedures
- Dishwasher safe*
- Care should be taken to ensure that surface temperatures of hot cabinets/plate warmers are below 100°C
- Although Medeci Systemware is extremely tough it is not scratch resistant and can, in normal use, be scratched relatively easily by the use of sharp cutlery.

*It is advised if cleaned in a dishwasher, they be placed on the top shelf and away from the heating element, and a lower temperature cycle selected.

Important:

- **DO NOT** Use in a conventional oven/grill
- **DO NOT** Use in a microwave oven/grill
- **DO NOT** Use abrasive cleaning or drying products



Medeci System Plate Range

- H79969 - Ivory
- H79982 - Yellow
- H80005 - Blue
- H80017 - Red



Medeci System Cup Range

- H69800 - Yellow
- H70097 - Blue
- H70103 - Red
- H70115 - Ivory
- F23938 - Medeci Spout Top
- F23914 - Medeci Anti spill Top

Medeci Plate

Medeci plates are designed to combine light weight and high strength and are appropriate for people who have difficulty eating from normal tableware. The plate has a raised outer lip, one handed eaters will find this particularly useful as it can be used to maintain the food steady while loading it onto a fork or a spoon, when used in conjunction with a non-slip table mat

Medeci System Cup

Medeci cups are a lightweight, high strength cup designed to offer considerable choice. People who find drinking from normal tableware difficult will find the Medeci cup easier to use. The Medeci systemware cup range also offers the choice of two tops;

- 1 – Anti Spill top
- 2 – Angled Spout Top

Medeci System Cup; Instructions for Use

Each Medeci cup comes with two handles packed separately. They can be fitted into the grooves around the outside of the cup. The position of the two handles is a matter of choice for the user. One or both of the handles can be used; if both handles are used they can be positioned in different ways to conform to the abilities of the users hands. (See System Cup Handling Options below)

Handle Removal

The handles can be removed from the groove by inverting the cup and tapping the handle sharply on a hard surface such as the edge of a table or work unit.

System Cup Handling Options

Two Handle fist grip



Two Handle open grip



Wide grip for one hand



To remove



Medeci Cup Tops

1 – Spout Top (Code: F23938)

The spout top has an angled spout for people who find it difficult to cope with conventional cups or cups with anti spillage tops. The spout is angled at 45 degrees to the horizontal when the cup is partly tipped. At this angle the spout may be used to depress the tongue to assist in control during drinking.

2 – Anti Spill Top (Code: F23914)

The anti spill top is designed primarily for people with tremor or poor hand control. It has four holes drilled in the top, three in one side and one on the opposite side, the user may choose whether to drink through the side with three holes or the side with one hole according to the flow rate desired. The primary purpose of the anti spill top is to contain the drink within the cup. This is a benefit for people who suffer from a tremor or have limited hand control, and hence are at risk of spilling or splashing the contents of the cup



Red cup
fitted with F23938 -
Spout Top



Yellow cup
fitted with F23914 -
Anti Spill Top

General Information

Please ensure these instructions are fully read and understood before use. Leave with the tableware or keep in a safe place for future reference.

Manufactured in the UK for:

NRS Healthcare

Clinitron House, Excelsior Road,
Ashby Park, Ashby de la Zouch,
Leicestershire, LE65 1JG
United Kingdom

Tel: 0845 120 4522

Int'l: +44 (0) 1530 418378

Fax: 0845 121 8112

Email: customerservice@nrs-uk.co.uk

This product is from a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range visit: www.nrs-uk.co.uk or request a catalogue from the above number.

www.nrs-uk.co.uk