

User Instructions

Telescopic Channel Ramp

TRP-207 Range



Product Code SKU	Length Minimum	Length Maximum	Usable Width	Overall Width	Product weight (pair of ramps)*	Maximum load height	Capacity
TRP207-4	745mm	1220mm	150mm	190mm	5.2kg	200mm	275kg
TRP207-5	895mm	1520mm	150mm	190mm	7kg	250mm	275kg
TRP207-6	1050mm	1830mm	150mm	190mm	8.3kg	300mm	275kg
TRP207-7	1200mm	2130mm	150mm	190mm	8.7kg	350mm	275kg

**Product weights are approximate and may vary*

General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

This Telescopic Channel Ramp, TRP207 is designed to provide access for manual wheelchair users* (and their carers, if appropriate) over thresholds, steps and curbs. The two part telescopic design is easily adjusted to provide the desired length. There are various available length versions in the range, see table above for size information

Unpacking

- No tools are required to assemble this product. (However if the ramp is to be secured to the landing surface, then appropriate tools will be required)
- Remove items from packaging; Check for damage. If you see any, or suspect a fault, do not assemble or use the product; contact your supplier immediately.

Safety Precautions

When choosing a ramp, the length, and therefore the angle of the ramp depend upon:

- The height to be raised
- The type of wheelchair or scooter
- The weight of the wheelchair or scooter and the user
- The strength of the helper or user.

As a general guide, a minimum gradient of 1 in 6 is suggested, although 1 in 8 is recommended, when the user is seated on the wheelchair or scooter.

For loading an unmanned wheelchair or scooter into a vehicle, a higher gradient can be used, but this should not exceed 1 in 3.3 In addition to this, it is important that you refer to your vehicle's user guide for the proper degree of incline/decline and direction before using the ramp.

- Never exceed its recommendations.

- Only carry the ramp using the carry handles. Take care when lifting; follow standard manual handling procedures.
- Take particular care when using the ramp in wet/damp conditions.
- *Ramp not suitable for powered wheelchair and/or mobility scooters.

Instructions for installation and use

1. Remove all parts from the packaging and check the ramps are free from damage.
2. Press the button and pull out the telescopic part until clicks into place, to extend the ramp, hold it horizontally or lay it on the floor. Note: If the latch is released (or is not engaged) when the ramp is not horizontal, the inner section may slide out under its own weight. At full extension, the latch will engage to prevent the inner ramp sliding out totally.
3. Place the ramp on the landing surface ensuring that the top lip is resting securely on the landing surface.



4. Position the ramps at the required distance apart. The front and rear wheels of the vehicle should be able to run up the ramp freely.



5. Ensure the top lip is secure before using the Telescopic Channel Ramps
6. After use, press the button and slide the sections back together until the latch engages. When locked, the ramps can be carried safely.
7. Do not exceed the maximum load.

Important Note:

To prevent the ramp from becoming dislodged during use, ensure the top lip overlaps the landing surface as far as possible.

The surface holding the top lip needs to be substantial; make sure that the surface and supporting structure is of sound construction.

Care and Maintenance

- Inspect ramp regularly for signs of wear or damage.
- Do not use if the ramp is damaged.
- Clean regularly, wipe over with a solution of mild detergent and water. Remove any grit that may have become caught between the sections.

Warranty

The Ramp People warrants that the ramp is free from defects in material and workmanship. This warranty shall remain in effect for 1 year from the date of original consumer purchase.

The warranty does not cover failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.

The Ramp People are not liable for incidental or consequential injury or damage to property.

Should it be necessary to return the ramp, it is essential it is adequately packed to protect it in transit, preferably in its original box; otherwise The Ramp People accept any responsibility for transit damage to the unit.

In some cases, the items will be supplied by 3rd party seller, please keep the original invoice.

This does not affect your statutory rights.

The Telescopic Channel Ramp Range are part of our range of ramps from The Ramp People. To view the full range please visit our website **www.theramppeople.co.uk** Or contact us via, post, email and/or telephone.

The Ramp People

Unit C3
Sandown Ind. Park
Mill Road
Esher
Surrey
KT10 8BL

Tel: +(44) 1372 478960

Email: hello@theramppeople.co.uk

www.theramppeople.co.uk