

Maximum User Weight



General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

1.0 Intended Use

The adjustable back support is intended to support a single user at a chosen inclined position whilst seated in bed. It is fitted on top of the mattress and underneath any supporting pillows.

2.0 Safety Precautions

Ensure the supporting arm is correctly fitted into the location points before leaning back onto back support.

Ensure bed is either fitted with a fixed headboard or that the bed is positioned against a wall, so that the back support cannot slip backwards during use.

Periodically check the stability and rigidity of the back support and, where applicable, that screws are tightened and in good condition.

3.0 Cleaning Instructions

Remove any dirt or dust using a soft, damp cloth. Use mild detergent solution to remove stubborn stains. Dry with a soft cloth. For disinfection in hospital environments, please follow hygiene specialist's recommendations.

NRS Healthcare

Sherwood House, Cartwright Way, Forest Business Park,
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK
Tel. Customer Services: +44 (0)345 121 8111
Email: customerservices@nrshealthcare.co.uk



4.0 Fitting Instructions

1. Put on top of mattress (and any relevant bedding), with rear edge of base against the headboard/wall.
2. Adjust angle of back support to desired position.
3. Place pillows against back support as required.
4. Test to ensure back support is firm and cannot move backwards.

The back support is then ready for use.

5.0 Product Guarantee

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS Healthcare is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk