

USER INSTRUCTIONS

SPA BATH STEP

P23099

SPA Bath Step

⚠ GENERAL SAFETY WARNING ⚠

Please make sure that you read and follow these instructions carefully. Failure to do so can result in injury. Keep these instructions in a safe place for future reference.

INTENDED USE

The SPA Bath Step is designed to assist users transferring into and out of the bath by reducing the step-over height. It can also provide assistance when using the toilet or sink in a bathroom environment.



SAFETY

The maximum user weight must not be greater than 25st / 159kg.

Make sure that all legs are in contact with the floor, to avoid rocking and instability.

Wipe the step dry after each use.

Take care when stepping off or onto the step to avoid slipping.

UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Contents

1. Bath Step x 1

ASSEMBLY

Assembly not required.

MAXIMUM USER WEIGHT



CLEANING

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

MAINTENANCE

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules and non-slip strips.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

ACCESSORIES AND SPARES

P57875: Replacement set of ferrules.

P57863: Replacement set of 10 non-slip strips.

PRODUCT DISPOSAL

Product should be thoroughly cleaned prior to disposal.

Product is manufactured from metal, plastic and bamboo and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

OTHER NRS PRODUCT IN THE SPA RANGE

P23075 - SPA Shower Stool



P42409 - SPA Perching Stool



NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.
Tel. Customer Services: 0345 121 8111
Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids, to view our full range, visit our website:

www.nrshealthcare.co.uk

P23099-UI-2