

This product is from a range of over 3000 daily living aids available from NRS Healthcare.

To view the full range please visit our website or contact us:

Tel: 0345 121 8111



User Instructions

Flat End Steel Grab Rail Range

**See table overleaf for a full list of products available*

**Maximum User
Weight**



General Safety Warning

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference.

Intended Use

This Flat End Grab Rail Range is recommended for all applications, internal and external, and are designed and tested to comply with:

BS EN 12182. They can be used on their own or as part of a group of products which combine and complement each other to provide a safer environment for the user.

Please refer to our website or catalogue for product detail.

See back page for contact details.



Unpacking

- Remove the product from its packaging.
- Check for damage.
- If you see, or suspect, a fault do not install the product
- Contact your supplier immediately.

NRS Healthcare,
Sherwood House,
Cartwright Way,
Forest Business Park,
Bardon Hill,
Coalville,
Leicestershire,
LE67 1UB,
United Kingdom.
Tel. Head Office: 0330 100 0098
Tel. Customer Services: 0345 121 8111
Email: customerservice@nrshealthcare.co.uk

www.nrshealthcare.co.uk

Safety Precautions

- read these instructions fully before installation and use.
- This Grab Rail must be correctly assembled, installed, and maintained.
- The product must be installed by a suitably qualified, competent person.
- Only install the product in an area where the mounting surface is structurally sound.
- Strengthening modifications to the fixing surface may be required.
- Only install this product with fixings matched to the grab rail, type of mounting surface and the loading conditions to be applied. NRS is unable to specify the type of fixing to use because of the vast number of possible types and surface condition to which the grab rail may be attached
- The maximum Load of this product is 160kg (25st) based on the product being fitted to a suitable surface using appropriate fixings and fitted by a competent person.

Installation General

- Before installation; consider the best place to position the rail, taking into account all activities it may be used for, e.g. sitting, standing and reaching.
- Grab Rails are intended to be used to provide gripping or handle positions around doorways, stairwells, bathrooms and for other similar applications.
- After the position of the rail is chosen for optimum use, consider the structure of the wall and avoid any electrical/plumbing behind the wall.
- Important; after installing the product, perform a final tighten and inspection of all fasteners.
- Test the installation by vigorously pulling on the rail. The rail should be rigid and stable.

Instructions for Use

- The grab rail is designed for general domestic use as an aid to the movement of a person about a normal dwelling.
- The grab rail must only be used for the tasks it is intended to perform.
- Do not use it to tie or hang objects from it or to secure items to it, as this may have serious detrimental effects on its performance and safety

Care & Maintenance

- Inspect on a regular basis the stability and rigidity of the product.
- Check all screws are tightened and in good condition
- Any rails showing signs of damage or misuse must be replaced.

Specification

This product is available in various size and types, for a full list of options, please refer to the table below

This product is designed for a single location installation.

Max* weight capacity:
160kg (25st)

* Weight capacity only refers to the rail, not to the fixings or retention in the mounting surface

Cleaning

- The grab rail should be cleaned periodically using normal domestic cleaning materials.
- After cleaning always rinse with water and wipe thoroughly with a damp cloth to remove any cleaning material residue.
- Do not use strong or concentrated acidic, alkaline or other cleaning materials which may damage or discolour the rail.
- Where disinfecting is required, use a mild detergent/disinfectant solution.
- Do not use abrasive cleaners

Packaging

The product is individually packed to prevent it from becoming soiled before use; it may also be supplied bulk-packed in an NRS cardboard carton. Both are suitable for recycling and should be disposed of in accordance with local recycling regulations

WHITE		BROWN	
CODE	SIZE/TYPE	CODE	SIZE/TYPE
M91231	305mm (12")		
M83933	455mm (18")	M83945	455mm (18")
M83957	610mm (24")	M83969	610mm (24")
M83970	710mm (28")	M83982	710mm (28")
M83994	455mm (18") Off-set	M84007	455mm (18") Off-set
M84019	610mm (24") Off-set	M84020	610mm (24") Off-set
M84032	710mm (28") Off-set	M84044	710mm (28") Off-set