

Introduction

The Days Healthcare 600 'Cardiff' legrests has been designed to align with seat heights varying from 38 to 51 cm. The independent spring button adjustment also allows the legrest to be tilted in either direction. Concave top, size 50 × 31 cm made from soft density foam, covered with PVC leather cloth. The base is made from chrome plated square section steel tubing and fitted with either brake-type castors which can be locked or rubber tips to assist the legrest remaining static.

Height Adjustment

To adjust the leg rest height push in the single spring button and pull the extension leg to its new position.

Maintenance

The Days Healthcare 600 legrests are designed for minimum maintenance. Manufactured from non-corroding material and require the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasive materials.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Do not apply full body weight on one side of the leg rest only.

Periodically check for elongated or stressed adjustment holes.

Check that the castors rotate and swivel freely and there are no debris building around the axles. Check rubber tips are intact and not severely worn or split.

Check that all fastenings are secured.

Warranty Information

The Days Healthcare 600 legrests are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

