

USER INSTRUCTIONS

MOWBRAY TOILET SEAT & FRAME RANGE

EXTRA WIDE

M48805	Mowbray Frame & Seat Extra Wide
M48817	Mowbray Frame & Seat Extra Wide – Floor Fixed

⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

To assist those who have difficulty in lowering onto or rising from a standard toilet. The comfortable plastic moulded seat offers support in a natural position. An optional floor fixing kit is also available (F11572).



2. SAFETY

WARNING: The maximum user weight must not be more than 254 kg (40 st).

DO NOT: Attempt to stand or climb on the product.

MAKE SURE: All legs are set at equal height to avoid rocking.

2.1 Floor Fixing Safety

It is recommended that a suitably qualified, competent person installs this product.

Always check for concealed electrical cables and water pipes when installing; also check your legal requirements relating to earth bonding of extraneous metal parts.

Inspect the product at regular intervals for wear and damage.

CAUTION: Never drill through pre-stressed concrete.

CAUTION: Use appropriate sized rawl plugs if this product is being fitted onto a concrete floor.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

Maximum User Weight



3.1 Contents:

Item	M48805	M48817
Toilet Seat	1	1
Seat Frame Assembly	1	1
Leg Extensions with Ferrules	4	n/a
Leg Extensions with Floor Fixings	n/a	4
E-clips	4	4

4. ASSEMBLY

These products are supplied fully assembled apart from the seat.

4.1 Fit the Seat

Remove the seat from the bag and snap into place on the centre bars of the frame.

5. INSTALLATION

Adjust the height for the toilet and user before installation. Only install the product in an area where the mounting surface and substrate are structurally sound. Strengthening modifications to the fixing surface may be required.

Also check the type of substrate you intend to fix it to e.g. brick/block, timber or concrete. Always check that the type of fixing you are going to use is compatible with the substrate.

Note: As the substrate can vary considerably, we cannot recommend any particular fixings; if in any doubt; consult a reputable fixings supplier for advice.

5.1 Adjust the Height

1. Remove the 4 x leg extension E-clips.
2. Slide the leg extensions to the required height, align the holes and replace the E-clips.
3. Lift the toilet seat and lid.
4. Place the Mowbray over the toilet with the cutaway of the seat at the front and the integral splash guard placed centrally over the toilet bowl.

Note: Make sure the seat and toilet frame are fully secure before use.


5.2 Securing the Feet to the Floor (M48817)

1. Insert the floor fixing legs into the toilet frame.
2. Slide the legs so the frame is at the required height for the user.

Note: Make sure that all legs have been adjusted to the same height. Height adjustment is not possible once the legs are secured to the floor.

3. Align the holes and replace the E-clips.

5.2.1 Mark the Hole Positions

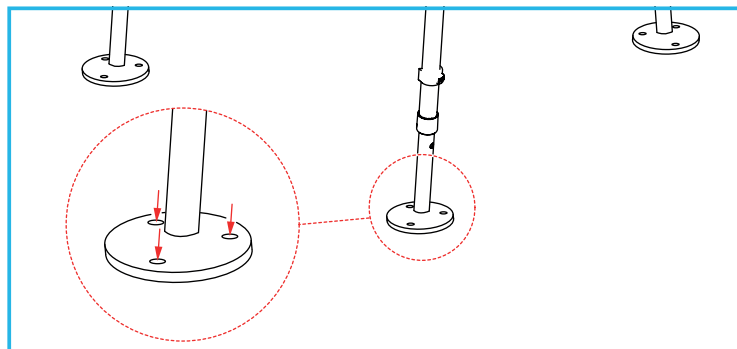


Place the toilet frame in the required location and mark the hole positions to be drilled.

5.2.2 Drill the Fixing Holes

1. Remove the toilet frame from the required location, then drill the holes in the positions marked, as appropriate for the chosen fixings.
2. If you are using rawl plugs for the installation, insert the rawl plugs into the fixing holes.

5.2.3 Fix the Feet to the Floor

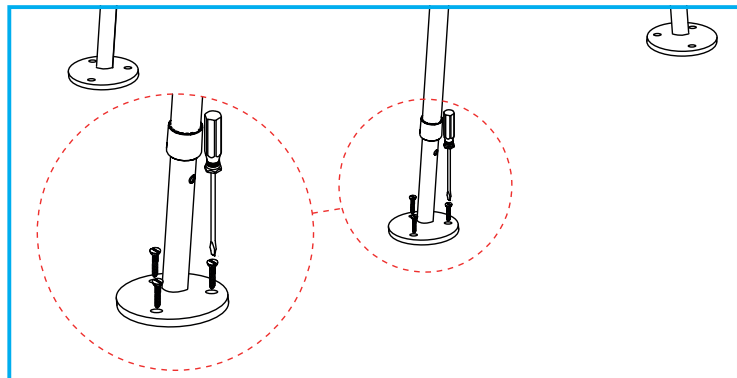

Using an appropriate tool for the chosen fixings, secure the floor fixing feet to the floor.

Note: Check that all fixings are fully secure before use.

5.3 Fitting the Optional Floor Fixing Kit to M48805

Please refer to the user instructions provided with the optional floor fixing kit (F11572).

6. MAINTENANCE

Check that all fixing clips are inserted correctly and that there is no excessive movement.

If a floor fixing kit has been fitted, check the fixing screws are secure.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

7. CLEANING INSTRUCTIONS

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

Regularly clean with a non-abrasive cleaner.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

This product is suitable for thermal decontamination and can be steam cleaned or autoclaved on cleaning cycles up to 70°C.

Allow product to fully dry prior to use/storage.

8. ACCESSORIES & SPARE PARTS

F11572 - Floor Fixing Kit

L84207 - Extra Wide Seat

For further information please visit our website or call Customer Services on the details below.

9. PRODUCT DISPOSAL

Thoroughly cleaned/sterilise this product prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

10. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park
Bardon Hill, Coalville, Leicestershire, LE67 1UB, UK.

Tel: Customer Services: +44 (0)345 121 8111

Email: customerservice@nrshealthcare.co.uk

NRS Healthcare is a leading supplier of community equipment and daily living aids.
Please visit one of our websites to view our full range of products:
www.healthcarepro.co.uk or www.completecareshop.co.uk