

PRODUCT TECHNICAL DATA SHEET

LINTON PLUS RAISED TOILET SEAT

| | | | |
|--------|------------------------------------|--------|-------------------------------------|
| M53163 | 2" (50mm) Non Retail Pack - No Lid | M53175 | 4" (100mm) Non Retail Pack - No Lid |
| M53564 | 2" (50mm) Retail - No Lid | P42469 | 4" (100mm) Retail - With Lid |
| P42458 | 2" (50mm) Retail - With Lid | P42446 | 6" (150mm) Retail - No Lid |
| M53576 | 4" (100mm) Retail - No Lid | P42471 | 6" (150mm) Retail - With Lid |

Product Description and Intended Purpose

The Linton Plus toilet seat's raised design makes sitting and standing from the toilet easier, with less strain on the user's joints. The cut-out seat design makes personal cleansing easier and distributes pressure more evenly away from the coccyx. The seat is adjustable and fixes securely to standard shaped toilets. Seats are available in 3 sizes and can be supplied with or without a lid.

Product Features

- Smooth and contoured surface to add comfort and aid lowering and rising.
- Designed with cut outs to help distribute pressure more evenly and allow easier personal cleansing.
- Durable lightweight plastic, easy to clean.
- Adjustable brackets to enable fit to most common toilet seat sizes.

Technical Specification and Dimensions

- Raise Height - Small 50mm (2")
- Raise Height - Medium 100mm (4")
- Raise Height - Large 150mm (6")
- Aperture - 270 x 225mm

Safety Precautions

- The maximum user weight must not be greater than 30st (190kg).
- Do not stand on the seat.
- Inspect the seat and fixings at regular intervals to make sure that it is safe and secure.
- Do not use the seat unless you are sure it is safe to do so. If in doubt, contact your supplier for advice.
- Inspect the seat and fixings at regular intervals for wear and damage.
- If damage is found, stop using the seat and contact your supplier immediately for advice.

Installation Information

- Refer to the User Instructions M53163-UI-6.

Cleaning

- Clean regularly, using a general purpose, neutral detergent and warm water.
- Please check the suitability of the product before use. Do not use abrasive cleaners or cloths as this will permanently damage the surface and invalidate any warranty.
- Do not use any form of solvent as this will damage the surface and weaken the product and invalidate any warranty.
- To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.
- These seats may be autoclaved on cleaning cycles up to 73°C.
- Dry the seat thoroughly before use.



Maintenance

- We recommend that you check the seat at regular intervals.
- Make sure that the screws are tight and the seat remains evenly positioned on the toilet bowl.
- If you think the seat has moved out of position, reposition in accordance with user instructions.
- If you remove the seat for any reason, it must be re-installed in accordance with the user instructions. Failure to do so could compromise the safety of the user.
- Check for general wear or damage. Any damage should immediately be reported to the supplier and the product taken out of action.

Accessories / Related Products

- None.

FAQ's

- None.

NRS HEALTHCARE Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville,
Leicestershire, LE67 1UB, UK.

Tel. Customer Services: 0345 121 8111

Email: customerservices@nrshealthcare.co.uk

NRS Healthcare is a Leading Supplier of Community Equipment and Daily Living Aids.
To view our full product range, visit our website:

www.nrshealthcare.co.uk

M53163-PTDS-2