

Product Technical Data Sheet 094 Elbow Crutches with Comfy Handle

Product Range

P23373	Regular - Pair
P23385	Long - Pair
L65808	Regular - 15 Pcs
F19042	Long - 15 Pcs



Product Description & Intended Purpose

Designed to provide additional support to people who have difficulty walking but are able to partially weight bear.

Product Features

- Ergonomic handles give superior pressure distribution and increased comfort.
- Height adjustable to suit a full range of users.
- Height adjustable arm cuff.

Technical Specification

Handle Height:

- **Regular** - 580 - 835mm (23" - 33")
- **Long** - 680 - 935mm (27" - 37")

Cuff to Handle: 200 - 275mm (8" - 10¾")

Maximum User Weight: 20st 127kg

Product Weight: approx. 2kg (4lb)

Installation Information: *(Note: for detailed information refer to User Instructions)*

- Product is packaged as a pair and requires no assembly prior to use.
- Height adjustment takes less than 5 minutes, see user instructions for details.

Safety Precautions

- Do not exceed the maximum user weight 20st, 127kg.
- Do not use if the crutches appear faulty or damaged.
- Do not use on wet, slippery floors or outdoors in adverse weather conditions.
- Do not hang items from the handles or cuffs as this will affect the stability of the crutch during use.
- Always plan the route to avoid obstacles before use. Take care on sloped surfaces, stairs and when crossing door thresholds and curbs etc.

PTDS094

Elbow Crutch with Comfy Handle

Cleaning

- Clean regularly using a general purpose, neutral detergent and warm water. Do not use bleach.
- Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.
- Allow the product to dry thoroughly prior to use/storage.

Care & Maintenance

- Check that all fixing clips are inserted correctly and there is no excessive movement.
- Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the tubes, adjustment holes and rubber ferrules.
- Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

Accessories / Related Products

- F17938 - Spare Ferrule - Grey
- M37364 - Crutch Pod

FAQ's

- How do I know which height to set the crutches at?

Stand upright in your normal footwear with your arms relaxed by your sides, the crutch handle should be in line with your wrist.

Product Supplied By:

NRS Healthcare Sherwood House, Cartwright Way, Forest Business Park, Bardon Hill, Coalville, Leicestershire. LE67 1UB UK