

105 x 147mm

Lifemax

Model **1084**

Medically Approved Hearing Aid


*Amplifies conversations, radio or TV sounds
Tone control helps with different types of hearing loss. Suitable for either ear*

Instruction manual

Introduction

Package Contents:

- Hearing Aid
- 2 Ear buds (L/R)
- 3 Batteries
- Carrying case
- User manual

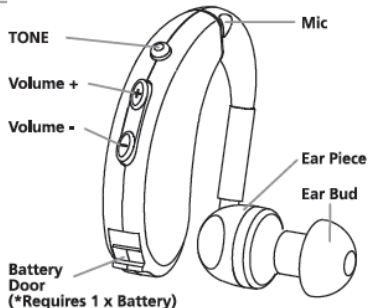

SPECIFICATION:

Operation Voltage: 1.4-1.5DCV
 Max. Current Drain: 6mA
 T.H.D.: 1KHz(ref.V): 2%
 Max. Output T.H.D.: 2KHz: 5%
 SIN Ratio: 54dB
 Output Power: 0.9mW
 Frequency Response:
 Low: 350Hz - 3KHz
 Medium: 500Hz - 5KHz
 High: 600Hz - 7KHz
 Wide: 700Hz - 11KHz

OPERATION CRITERIA:

Temperature Range: 5°C - 40°C
 Humidity: ≤80%
 Air Pressure Range: 86kPa - 106kPa

How to use

1. Selecting custom-fit Ear Bud


Included are 2 different size ear buds. Before use, choose from either L or M size to attach to the ear piece as shown in Pic#1. It is important to choose the appropriate ear bud size for maximum comfort & optimal sound quality.

Caution: Never insert the Ear Bud alone in your ear, it may be difficult to remove from the ear if it is not attached to the device.

2. Inserting battery

- Remove protective tab from the Zinc - Air battery before use.
- Open the battery door in a direction as shown in Pic#2.
- Insert battery as indicated by the polarity symbols in the compartment as shown in Pic#3.
- Close the battery door.

Caution: For optimal performance, we recommended using new battery. When the level of amplification becomes poor or distorted, be sure that the Ear Bud is clear of wax or debris; if the amplification is still distorted, replace with new battery.

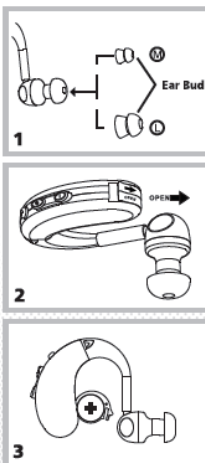

- Sound volume was preset at level 4 and Tone at "Medium" as shown in Pic#4.
- Wear the device in your ear by placing the Ear Bud in the ear & positioning the Ear Piece comfortably over the ear. To facilitate insertion of Ear Bud, lightly twist the unit left and right to achieve a snug fit. There is no need to push it into your ear strongly as shown in Pic#5.

Caution: If the Ear Bud does not fit your ear and whistling occurs (acoustic feedback), replace it with a different size of Ear Bud.


- Adjust the volume to the desired listening level (7- level selection: 1 - 7) and press "Tone" button to select among 4 Tones (Medium / High / Wide / Low) according to your personal hearing need as shown in Pic#6.
- Turn the unit OFF when not in use by adjusting volume to minimum (level 0) or opening the battery door to the lock hinge as shown in Pic#7.

Caution: If the unit will not be used for a long period of time, remove the battery from the unit.

7. Cleaning your Hearing Aid

We recommend that you clean your hearing aid regularly (e.g. twice a week) to remove any earwax that may build up as a result of regular use with a cotton tip or brush.

Caution: The device is not a water-proof product. Never immerse it completely in water <*IEC 529 (ECC) level 0*> It is suggested to place the device back to the carrying case after daily usage and well store it at room temperature with good ventilation and humidity ≤ 80%, and avoid contact with corrosive gases.

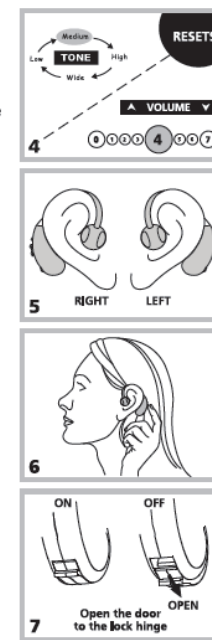

BATTERY INFORMATION

- This product uses 1 x LR44 or G13A alkaline button cell battery
 - Open the battery door as shown in Pic#2.
 - Remove the old battery and insert 1 x LR44 or G13A new battery. Observe polarity (+ -) sign when installing battery as shown in Pic#3.
 - Close the battery door.
- Use only designated battery type. Do not use re-chargeable batteries. Do not mix battery types.
- Read instructions carefully and install batteries correctly observing the polarity (+ -) sign to avoid leakage.
- Always replace sets of batteries at the same time. Do not mix old and new batteries.
- Do not attempt to recharge non-rechargeable batteries.
- Remove batteries from the product when it will not be used for an extended period of time to avoid leakage.
- Remove exhausted battery to avoid leakage.
- Do not short out battery contacts.
- Do not dispose of batteries in fire.
- Do not immerse electrically operated products in water or other liquid. Wipe clean only.

Please retain these instructions for future reference.

LIFEMAX LTD.
 Unit 1 Pipers Court,
 Thatcham,
 Berkshire, RG19 4ER
www.lifemaxuk.co.uk

For reasons of health and hygiene, this item is non-returnable.
 Your statutory rights are unaffected.

HAPI-030D AF2010-12-01
 Foshan Shunde Aiford Electronics Co., Ltd.
 Xinjiao Industrial Park, Daliang, Shunde,
 Foshan City, Guangdong Province, China
 2010-12

EC | REP | Shanghai International Trading Corp, GmbH (Hamburg)
 Eiffelstrasse 80, 20537 Hamburg, Germany

